

Transfer Tubes

The LumiraDx Platform is a smart, highly portable Platform that was designed to transform near patient testing. For blood-based assays, near patient testing is most efficiently achieved by direct fingerstick sampling, as opposed to needing to perform a venous blood draw.

In certain clinical situations, it may not be possible to perform direct fingerstick sampling. In these instances, a Transfer Tube can be used. Specific instructions for using a Transfer Tube from a fingerstick sample can be found in each assay-specific Product Insert at www.lumiradx.com.

This Technical Bulletin will aim to provide:

- Required Transfer Tube sample volume for each LumiraDx blood-based assay
- Required Transfer Tube anticoagulants for each LumiraDx blood-based assay
- Ordering information for Transfer Tubes that are available from LumiraDx

LumiraDx Assay	Required Sample Volume	Required Transfer Tube Anticoagulant	Transfer Tubes available from LumiraDx
INR	8 µL	Non-anticoagulated	L002170101050: Sarstedt Minivette – 10µL (Non-anticoagulated - 50 units/pack) or L002170201050: Sarstedt Minivette - 20 µL (Non-anticoagulated - 50 units/pack) or LUM-061665-48: KABE 20 µL (Non-anticoagulated - 48 units/pack)
D-Dimer	15 µL	Lithium Heparin	L002170301050: Sarstedt Minivette - 20 µL (Lithium Heparin - 50 units/pack) or LUM-061666-48: KABE 20 µL (Lithium Heparin - 48 units/pack)
CRP	20 µL	Lithium Heparin	L002170301050: Sarstedt Minivette - 20 µL (Lithium Heparin - 50 units/pack) or LUM-061666-48: KABE 20 µL (Lithium Heparin - 48 units/pack)
NT-proBNP	20 µL	Lithium Heparin	L002170301050: Sarstedt Minivette - 20 µL (Lithium Heparin - 50 units/pack) or LUM-061666-48: KABE 20 µL (Lithium Heparin - 48 units/pack)
SARS-CoV-2 Ab	20 µL	Non-anticoagulated	L002170201050: Sarstedt Minivette - 20 µL (Non-anticoagulated - 50 units/pack) or LUM-061665-48: KABE 20 µL (Non-anticoagulated - 48 units/pack)

While LumiraDx offer Transfer Tube solutions that can be ordered directly from LumiraDx, we also encourage local sourcing of Transfer Tubes, as long as the above general anticoagulant and volume requirements are met.

For product enquiries and technical support please contact LumiraDx Customer Services.

Email: customerservices@lumiradx.com

Phone: **+44 (0)1172841222**

www.lumiradx.com

Copyright © 2023 LumiraDx UK LTD. All rights reserved worldwide.

LumiraDx and Flame logo are trademarks of LumiraDx International LTD. Full details of these and other registrations of LumiraDx can be found at lumiradx.com/IP. All other trademarks are the property of their respective owners.

Content should be used for the use of the LumiraDx products only and in line with instructions provided. You may not, except with our express written permission, distribute or commercially exploit the content. Nor may you transmit it or store it in any other form of electronic retrieval system other than for the purpose of use of the LumiraDx Instrument or LumiraDx Test Strips. Information provided is subject to change without notice.

Products not available in all countries and regions. Please check with your local LumiraDx sales representative or distributor for availability in specific markets.

Manufactured by:

LumiraDx UK Ltd
Dumyat Business Park
Alloa
FK10 2PB
UK
Registration Number: 09206123

Authorized representative in the European Union:

LumiraDx AB,
Västra Vägen 5A,
16961 Solna, Sweden