


 LumiraDx™ Platform User Manual

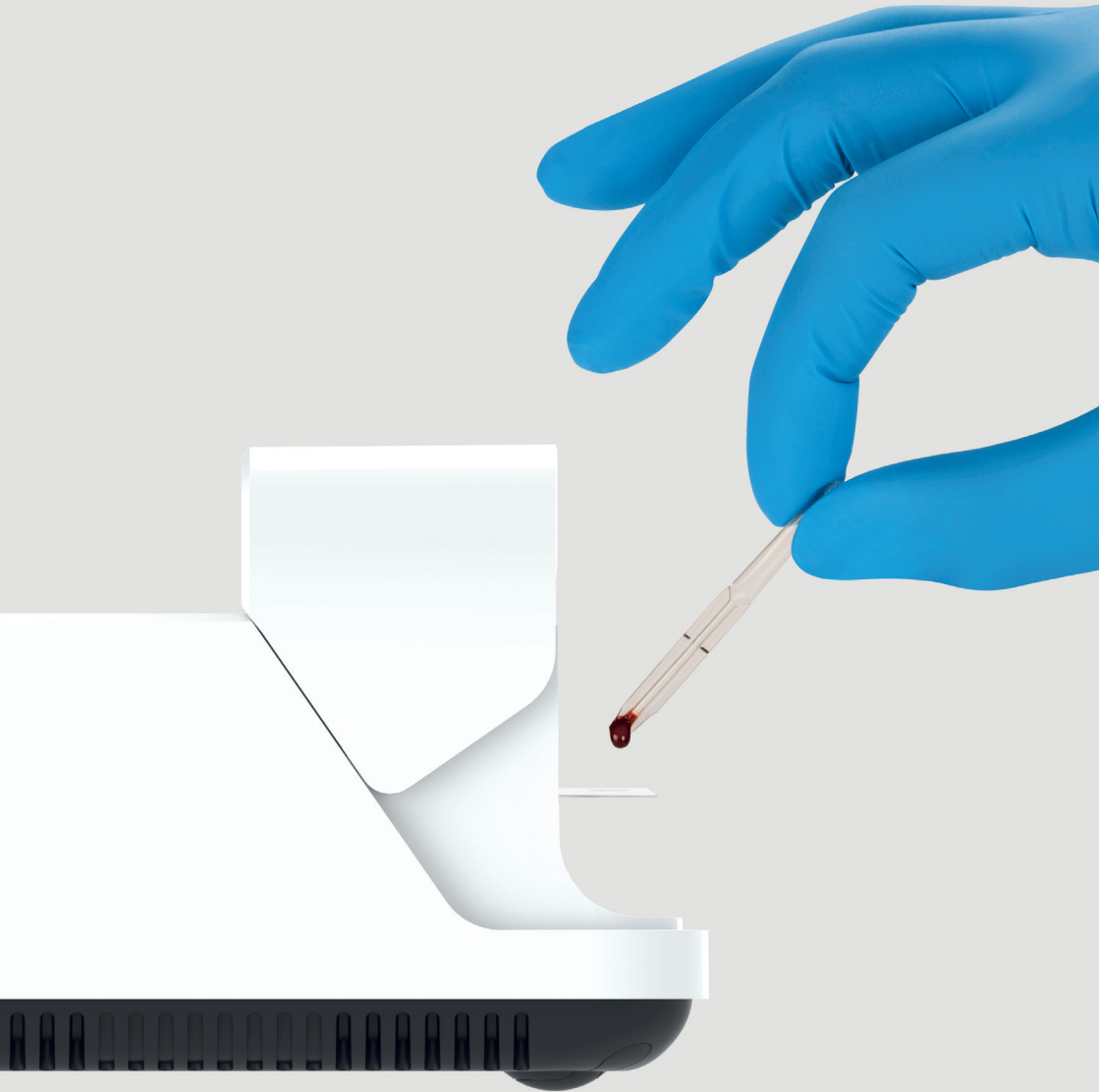

Contents

Contents

Symbols	5
Instrument Icons and Buttons	6
Abbreviations	7
Important Safety Information	8
Overview of the LumiraDx Platform	10
About this Platform User Manual	11
1 Introduction	12
1.1 Intended Use	12
1.2 Important Information	12
1.3 Summary of the Test Procedure	12
1.4 Principles of Operation	13
1.5 Storage and Operating Conditions	14
1.6 Warnings	15
1.7 Precautions	17
1.8 Help and Support	18
2 Getting Started	19
2.1 Unpacking	19
2.2 Powering the Instrument On and Off	19
2.3 Instrument Self-Check	19
2.4 User ID	20
2.5 Operating Mode and First Time Setup	20
2.6 The LumiraDx Instrument	21
2.7 Power Supply	22
2.8 LumiraDx Lot Calibration File Installation	23
2.9 Extended Functionality and Operating Modes	24
3 Preparation for Testing	25
3.1 Handling Test Strips	25
3.2 Inserting and Removing Test Strips	25
3.3 Sample Collection and Application	27
3.4 Applying a Sample to the Test Strip	27
3.5 Quality Control Solution Application	28
4 Instrument Operation	29
4.1 Home Screen	29
4.2 Settings Menu	30
4.3 Standalone Instrument First Time Setup	32
4.4 Connected Instrument First Time Setup	34
4.5 Display and Sound Preferences	36
4.6 Test Management	37

4.7	Performing a Patient Test	42
4.8	Quality Control	53
4.9	Performing a Quality Control Test	55
4.10	Result History	65
4.11	Onboard Quality Control	68
5	Cleaning and Disinfecting	70
5.1	Cleaning Procedure	72
5.2	Disinfecting Procedure	72
6	Software Updates	74
6.1	Check Software Version	74
6.2	Performing a Software Update	76
7	Troubleshooting	78
8	Maintenance and Disposal	82
9	Instrument Specifications	83
10	Ancillary Devices	84
10.1	LumiraDx Barcode Scanner	84
10.2	LumiraDx Printer	89
10.3	LumiraDx Connect Hub	93
10.4	LumiraDx Connect App	93
10.5	LumiraDx Platform Instrument Base	94
11	Customer Services	95
11.1	Warranty	95
12	References	96
13	Compliance	97
13.1	Environmental Practices	97
13.2	Compliance	97












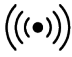





Platform User Manual

- Please note: Instructions are combined with example screenshots. Some screens may look different on the Instrument depending on the test or mode of operation. All screens, test names and results displayed in this Platform User Manual are intended only as examples.
- Refer to lumiradx.com/instrument for the most recent version of the Platform User Manual.
- For *In Vitro* Diagnostic Use





















Symbols

On the packaging and on the identification plate of the Instrument you may encounter the following symbols, shown here with their meaning:

	Temperature limitation
	Manufacturer
	<i>In vitro</i> diagnostic medical device
	This product contains electrical and electronic components that may contain materials which, if disposed with general waste, could be damaging to the environment. Residents of the European Union must follow specific disposal or recycling instructions for this product. Residents outside the European Union must dispose or recycle this product in accordance with local laws or regulations that apply
	Catalogue Number
	Serial Number
	"CE Mark". The product is in compliance with all relevant European Directives and Regulations. Refer to Declaration of Conformity for full details
	Caution - Consult Instructions for Use. Refer to safety-related warnings and precaution notes in the Instructions for Use accompanying this product, e.g. Platform User Manual, Test Strip Product Insert or Quality Control Product Insert
	Indicates that the Instrument can be potentially infectious due to the samples or reagents used
	Direct current - Indicates that the Instrument is suitable for direct current only and to identify relevant terminals
	Universal Serial Bus (USB) Port
	Near Field Connectivity (NFC) - Indicates the presence of the Radio Frequency (RFID) reader
	Identifies the power button to power on or power off the Instrument
	Consult Instructions for Use
	Authorized representative in the European Community
	Importer
	Unique Device Identifier

Instrument Icons and Buttons

The icons and buttons that appear on the touch-screen during normal operation of the LumiraDx Instrument are shown here, along with their respective meanings.

	Indicates the patient's biological sample
	Return to the home screen
	Info button - used to reveal additional information, such as test or patient information
	Indicates which fields barcode scanning is available as a means of data input. When pressed as a button, displays barcode scanning instructions
	Indicates when the Instrument is connected to Connect Manager
	Indicates that the Instrument has an active connection to an Electronic Health Record (EHR).
	Indicates that the Instrument connection to an Electronic Health Record (EHR) is not active
	Notification - Highlights an area which requires attention
	Caution - Highlights an error or precautionary statement which requires attention. Refer to safety-related warnings and precaution notes in the Instructions for Use accompanying this product, e.g. Platform User Manual, Test Strip Product Insert or Quality Control Product Insert
	Indicates the status of the battery
	Displayed when the battery is in a charging state
	Icon which acts as a button - clears the text input field of all data
	Indicates the high end of the touch-screen brightness slider
	Indicates the low end of the touch-screen brightness slider
	Volume (high)
	Volume (low)
	Indicates a text field which can perform a search function
	Indicates confirmation of a completed action

Abbreviations

CDC	Centers for Disease Control and Prevention
EHR	Electronic Health Record
HCP	Health Care Professional
ID	Identification
QC	Quality Control
RFID	Radio-Frequency Identification
UK	United Kingdom
USB	Universal Serial Bus
WHO	World Health Organization
HIS	Hospital Information System
LIS	Laboratory Information System
DOB	Date of Birth
RF	Radio Frequency
RH	Relative Humidity

Important Safety Information

Healthcare professionals need to adhere to standard precautions when using the LumiraDx Platform^{1,2,3}. All parts of the LumiraDx Instrument and LumiraDx Test Strips should be considered potentially infectious after use and are capable of transmitting pathogens between patients and healthcare professionals⁴.

The Instrument should be disinfected at least once per day when in use or if contamination is suspected, unless recommended otherwise for specific tests⁴. The Instrument may only be used for testing multiple patients when standard precautions and the manufacturer's disinfection procedures are followed. Full cleaning and disinfecting procedures are described in the "Cleaning and Disinfecting" chapter in this Platform User Manual.

A clean pair of gloves should be worn before testing each patient.

Wash hands thoroughly with soap and water before putting on a new pair of gloves and performing the next patient test.

Dispose of all Test Strips used for patient or Quality Control testing safely in accordance with local regulations and procedures.

The LumiraDx Instrument contains 2 USB ports that are for use with approved & supported USB devices only. DO NOT connect any unsupported devices into the USB ports.



The LumiraDx Instrument contains a neodymium magnet. Although the risk of interference with implanted devices such as pacemakers is minimal, we recommend that patients with an implanted device should maintain a distance of at least 15 cm between the LumiraDx Instrument and their heart device.



This User Manual and its content is copyright of LumiraDx Group Limited, 2023.

© All rights reserved, worldwide. Content should be used for use of the LumiraDx Products only and in line with instructions provided. You may not, except with our express written permission, distribute or commercially exploit the content. Nor may you transmit it or store it in any other form of electronic retrieval system other than for the purpose of use of the LumiraDx Instrument or LumiraDx Test Strips.

LumiraDx and Flame logo are protected trademarks of LumiraDx International LTD. Full details of these and other registrations of LumiraDx can be found at lumiradx.com/IP. All other trademarks are the property of their respective owners.



Overview of the LumiraDx Platform

The LumiraDx Platform is a point of care system which is used for *in vitro* diagnostic tests. It comprises a portable LumiraDx Instrument and a LumiraDx Test Strip for the required test. This allows healthcare professionals to perform tests using small sample volumes and to view results quickly on the Instrument touch-screen. Information about test duration and test performance can be found in the LumiraDx Test Strip Product Inserts.

The LumiraDx Instrument is intended to be for multi patient use.

The LumiraDx Instrument can be used in Standalone mode, providing test results on the touch-screen. Single or multiple Instruments can be connected to the LumiraDx Connect Manager for extended functionality and configuration. LumiraDx EHR Connect can enable the transfer of patient test results to the Electronic Health Record (EHR).

The LumiraDx Platform comprises:

Contained within this packaging	Available separately
<ul style="list-style-type: none"> • LumiraDx Instrument 	<ul style="list-style-type: none"> • LumiraDx Test Strips • LumiraDx Quality Controls • LumiraDx Barcode Scanner • LumiraDx Desktop stand for Barcode Scanner • LumiraDx Printer • LumiraDx Instrument Carry Case • Disinfectant Material – please refer to LumiraDx website for recommended disinfectant materials

LumiraDx Connect diagnostic connectivity solution:

- LumiraDx Connect Manager
 - LumiraDx Connect App (for iOS and Android)
 - LumiraDx Connect Hub
- LumiraDx EHR Connect

The mode of operation and selection of Platform components is dependent on the following requirements of the organization:

- Types of tests to be performed
- Number of testing sites and users
- System administration, integration, and data transfer to the EHR

About this Platform User Manual

Read this LumiraDx Platform User Manual and the LumiraDx Test Strip Product Inserts carefully and completely before testing for the first time.

This Platform User Manual provides the information required to operate and care for the LumiraDx Instrument.

If error messages appear on the Instrument touch-screen, follow the instructions displayed or refer to the "Troubleshooting" chapter. For questions not answered in the LumiraDx Platform User Manual or Product Inserts, please contact Customer Services. Refer to the "Customer Services" chapter of this Platform User Manual for contact information.

Please note: Instructions are combined with example screenshots. Some screens may look different on the Instrument depending on the test or mode of operation. All screens, test names and results displayed in this Platform User Manual are intended only as examples.

This Platform User Manual highlights cautions and important information:

This symbol indicates a caution. Refer to safety-related warnings and precaution notes in the Instructions for Use accompanying this product, e.g. Platform User Manual, Test Strip Product Insert or Quality Control Product Insert



1 Introduction

1.1 Intended Use

The LumiraDx Instrument (hereafter referred to as Instrument) is intended for use with the LumiraDx family of Test Strips (hereafter referred to as Test Strips) for the *in vitro* measurement of various analytes in a range of biological samples by trained healthcare professionals.

1.2 Important Information

Read this LumiraDx Platform User Manual and the LumiraDx Test Strip Product Inserts carefully and completely before testing. Pay attention to the "Important Safety Information" at the start of this Platform User Manual before operating the Instrument.

1.3 Summary of the Test Procedure

To perform a patient test, the user is required to insert a Test Strip, apply a small sample volume of one drop, and close the Instrument door. Patient ID can be entered using the touch-screen keyboard or LumiraDx Barcode Scanner.

When the test process is complete, the result is displayed on the touch-screen and the user can optionally add a comment. Upon completion of the test, the Instrument stores the test result.

The Instrument provides visual and audible prompts throughout the test process.

1.4 Principles of Operation

The Instrument automatically processes the Test Strip including sample movement, reagent mixing, thermal control and fluorescent reading of the reaction product and provides a calibrated, quantitative or qualitative result. Each Lot of Test Strips requires a LumiraDx Lot Calibration File, which provides the Instrument with the information required to process the test. The Lot Calibration File is installed by placing the RFID tag in the Test Strip Carton against the RFID reader in the Instrument.

1.4.1 Quality Assurance

The LumiraDx Instrument ensures the quality of test results obtained through the following features:

- Automated checks of the correct functioning of the Instrument at power on and during operation.
 - This includes electrical component operation, heater operation, battery charge state, mechanical actuators and sensors and optical system performance.
- Monitoring of Test Strip performance and controls during test runtime.
- Onboard Quality Control (OBC) assay for some tests (see Product Insert).
- Ability to perform Quality Control Tests using LumiraDx Quality Controls to meet regulatory compliance requirements.

1.5 Storage and Operating Conditions

1.5.1 LumiraDx Test Strips

For specific information on Test Strip storage and operating conditions, refer to the relevant LumiraDx Test Strip Product Inserts.

1.5.2 LumiraDx Instrument

The Instrument is portable and can be used across a range of settings, such as care homes, primary care clinics, hospital wards and departments, pharmacy and laboratories. The Instrument is not intended for hand-held operation.

The Instrument can be stored or transported at a temperature between -10°C and 50°C (14°F and 122°F).

To power on and access the Instrument result history, operate the LumiraDx Instrument at a temperature between 15°C and 30°C (59°F and 86°F), and at a relative humidity (RH) between 10 % and 90 % (non-condensing). Specific test operating conditions may be more restrictive. Allow time for the Instrument to acclimatize to operating temperatures if it has been stored in a cold environment.

Always place the Instrument on a level, stable surface when powering up or performing a test. Testing should be performed in a location where the air vents located in the back of the Instrument will not be blocked, i.e. on a table top away from soft surfaces. The Instrument will display an error message if these operating conditions are not met.

Avoid placing the Instrument in direct sunlight. This may interfere with proper functioning of the Instrument.

Refer to the "Instrument Specifications" chapter of this Platform User Manual for additional information on operating conditions and complete Instrument specifications.

1.6 Warnings

Inserting Test Strips



- **Do NOT** touch Test Strip Sample Application Area.
- **Do NOT** insert Test Strip when Instrument is powered off.
- **Do NOT** bend or fold the Test Strip.
- **Do NOT** touch Test Strip contacts.
- **Do NOT** apply sample until prompted.

Performing a patient test



- Wear a clean pair of gloves before performing a patient test.
- Place the Instrument on a level, stable surface before performing a test.
- Apply sample **AFTER** inserting the Test Strip and the Instrument prompt.
- **Do NOT** apply more than one drop of sample.
- Close the Instrument door after applying the sample and the Instrument prompt.
- **Do NOT** touch the Test Strip until the test has finished and the result is displayed.
- **Do NOT** open the door or move the Instrument during the test or an error will result and the test result will not be saved.
- Dispose of all Test Strips used for patient testing safely in accordance with local regulations and procedures.
- Follow the test operating conditions in the LumiraDx Test Strip Product Insert.
- Follow the information on correct handling of Test Strips in the LumiraDx Test Strip Product Insert.

Performing a Quality Control test



- Place the Instrument on a level, stable surface before starting a test.
- Use only LumiraDx Quality Controls.
- Apply Quality Control solution **AFTER** inserting the Test Strip and the Instrument prompt.
- **Do NOT** apply more than one drop of Quality Control solution.
- Close the Instrument door after applying the Quality Control solution and the Instrument prompt.
- **Do NOT** touch the Test Strip until the test has finished and the result is displayed.
- **Do NOT** open the door or move the Instrument during the test or an error will result and the test result will not be saved.
- Dispose of all Test Strips used for Quality Control testing safely in accordance with local regulations and procedures.
- Follow the test operating conditions in the LumiraDx Test Strip Product Insert.
- Follow the information on correct handling of Test Strips in the LumiraDx Test Strip Product Insert.

Cleaning and disinfecting



- Always wear gloves whilst cleaning and disinfecting the Instrument.
- Always use LumiraDx recommended materials to disinfect the Instrument. Details of LumiraDx recommended materials can be found at lumiradx.com.
- **CAUTION: Excessive liquid may damage the Instrument.** It is important for the protection of the Instrument that exposure to excess moisture is prevented. All disinfection cloths and/or wipes should only be slightly damp, with any excess liquid being manually removed from the cloth or wipe before use.
- **CAUTION:** Liquid should not be placed in or near the vents on the bottom or back of the Instrument.
- **CAUTION:** Alcohol wipes alone are not sufficient to disinfect the Instrument for blood-based samples, due to the potential presence of bloodborne pathogens.
- **Do NOT** spray or pour solution directly onto the Instrument.
- Following cleaning and disinfection, and before performing a patient test, change gloves and wash hands.

Instrument disposal

- **Do NOT** attempt to replace or reinstate the battery.



1.7 Precautions

Only operate the LumiraDx Instrument for its intended purpose and in accordance with this Platform User Manual and warnings. If the Instrument is used in a manner not specified in the Platform User Manual, protection provided by the equipment will be impaired. The LumiraDx Instrument (including power supply unit) is designed to operate within the manufacturer specifications. Do not exceed the manufacturer specifications when in use.

Avoidance of electrical shock, fire and explosions

- Use only the power supply unit that is provided with the Instrument. Use the appropriate power blade for the region. The correct power supply unit and adapter is required to maintain the safety and electromagnetic compatibility of the system.
- Always operate this Instrument on a clean, level and stable surface. Do not drop the Instrument. Ensure that air flow to the ventilation openings located on the back of the Instrument are not restricted.
- Risk of electrical shock. Do not operate the Instrument or the power supply unit if it has been opened, damaged or exposed to moisture, condensation or rain.
- Overheating can cause the battery pack to catch fire or explode.



Safe disposal



- Follow proper infection control guidelines for handling all specimens and related items. Properly dispose of all contaminated waste according to local regulations.
- The LumiraDx Instrument and its components must be treated as potentially biohazardous waste. Decontamination (i.e. a combination of processes including cleaning, disinfection and/or sterilization) is required before reuse, recycling, or disposal.
- Dispose of the system or its components according to the appropriate local regulations.

Electromagnetic interference



- Do not use the Instrument near strong electromagnetic fields, which could interfere with the proper operation of the Instrument.

Electromagnetic compatibility



- This Instrument has been designed and tested to CISPR 11 Class A. In a domestic environment it may cause radio interference, in which case, you may need to take measures to mitigate the interference.

1.8 Help and Support

Information about using the LumiraDx Instrument can be found in this Platform User Manual. Information about the Test Strips and test performance can be found in the LumiraDx Test Strip Product Inserts and Quality Control Product Inserts. Information about LumiraDx Connect Manager and LumiraDx EHR Connect can be found in the LumiraDx Connect User Manual.

The documents can also be found at lumiradx.com/instrument.

If error messages appear on the screen, refer to the “Troubleshooting” chapter in this Platform User Manual. For questions not answered in the LumiraDx User Manuals or Product Inserts, please contact Customer Services. Refer to the “Customer Services” chapter of this Platform User Manual for contact information.

2 Getting Started

2.1 Unpacking

The LumiraDx Instrument package includes the following contents:

1. LumiraDx Instrument
2. LumiraDx Power Supply Unit
3. Platform User Manual
4. Platform Quick Reference Guide (including passwords for Standalone operation)

Inspect the Instrument and packaging for damage before use. Report any damage to Customer Services. Refer to the "Customer Services" chapter of this Platform User Manual for contact information.

2.2 Powering the Instrument On and Off

The Instrument must be fully charged before first use. It will take approximately 2 hours to fully charge the battery using the power supply unit provided.

Place the Instrument on a level, stable surface. Power on by briefly pressing the power button at the rear of the Instrument. You will hear the Instrument powering on and the display will be blank for several seconds. When prompted enter user ID and/or password to login.

The Instrument touch-screen will dim after 2 minutes of inactivity. Tap the touch-screen to restore the touch-screen brightness.

To power the Instrument off after use, press the power button at the rear of the Instrument for 2 seconds, and tap the screen message to confirm power off. The Instrument should be powered off for transportation and when not in use.

2.3 Instrument Self-Check

The Instrument will perform a self-check when starting up. Ensure that the Instrument is on a level, stable surface during this process. If the Instrument door is open, the Instrument will prompt to close the door before performing the self-check. If a Test Strip is inserted at this time, the Instrument will prompt to remove the Test Strip and close the door. The self-check will not proceed until the door is closed.

2.4 User ID

There are two types of identification used with the Instrument:

General user

Allows general users to use the Instrument to perform patient and Quality Control tests, view result history, and adjust preferences including screen brightness and sounds.

Administrator

The administrator has additional rights to Instrument settings including language, time and date if the Instrument is being operated in standalone mode. If the Instrument is being managed in a connected mode then these settings will be downloaded from Connect Manager.



2.5 Operating Mode and First Time Setup

The language and current date and time will need to be set by the administrator prior to performing a patient test. Other sound and display preferences can be set by any user.

Settings will automatically be downloaded to the Instrument if connecting to LumiraDx Connect Manager. Refer to the "Instrument operation" chapter of this Platform User Manual for more information on first time setup.

2.6 The LumiraDx Instrument

The Instrument is used in conjunction with the LumiraDx family of Test Strips for the *in vitro* quantification of various analytes in a range of biological samples.

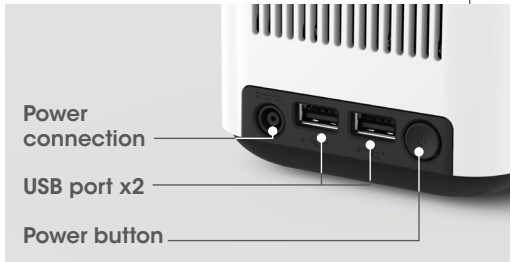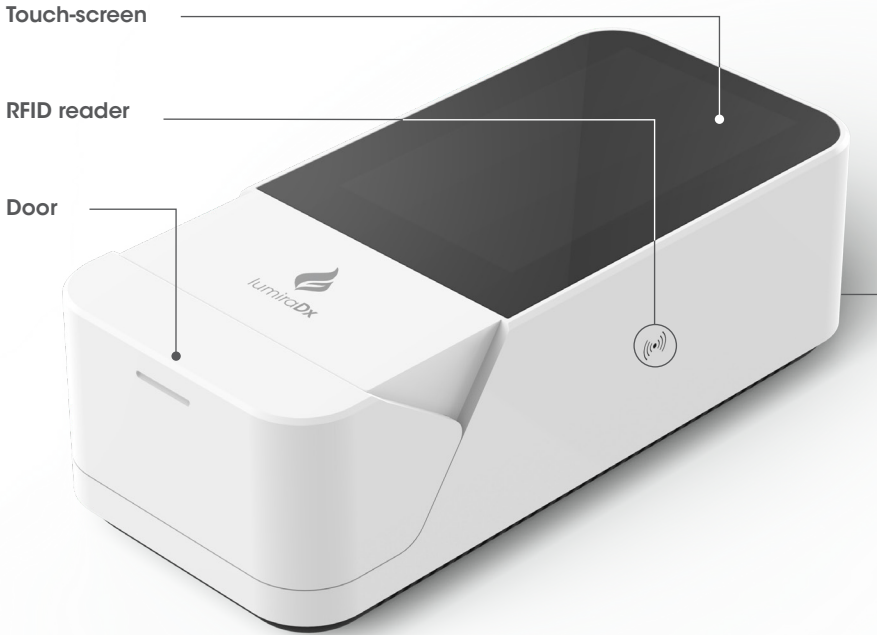

2.7 Power Supply

The LumiraDx Instrument contains a rechargeable non-removable battery. If the battery is critically low, the Instrument will prompt to connect the power supply unit. It will take approximately 2 hours to fully charge the battery. The Instrument can be operated while charging the battery.



During operation, the Instrument touch-screen always displays the battery power level and provides an alert when the battery level is low. To save power when battery operated, the Instrument will automatically power off after 55 minutes, unless the touch-screen is tapped. When the Instrument powers off, all result records will remain in the Instrument memory and any Instrument settings will be maintained.

Refer to the "Instrument Specifications" chapter of this Platform User Manual for additional information on operating conditions and complete Instrument specifications.

2.8 LumiraDx Lot Calibration File Installation

LumiraDx Lot Calibration Files are required to provide the Instrument with the information needed to perform diagnostic tests. Every manufactured Lot of Test Strips has a unique Lot Calibration File. This contains information about the test method, the Lot number, the calibration data and expiration date.


The Instrument will prompt the user to install the Lot Calibration File only when a new Test Strip Lot is used. If no Lot Calibration is required then you can proceed with the test normally.

When prompted, click the manual install button. Locate the ((Ⓢ)) symbol on the side of the Instrument. Then place and hold the back of the Test Strip Carton to the ((Ⓢ)) symbol for 3 seconds. When the Lot Calibration File is installed, a message will appear on the screen. The Instrument will also make a sound (if sounds are turned on in the Instrument settings). Once installed, the Instrument will be able to run this and any future tests from the same Lot of Test Strips. This process will need to be repeated when a new Lot of Test Strips is used.

Alternatively ahead of time, in Administrator mode Lot Calibration Files can be loaded on to the Instrument so that they are available for use. Select 'Settings', 'Test Management' then 'Install Lot Calibration File'. The Instrument will guide through the steps that are required.


Prompt to install

The Instrument will prompt the user to install the Lot Calibration File if a new Test Strip Lot is used.

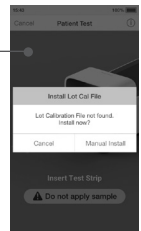

RFID reader

Locate ((Ⓢ)) symbol on Instrument.

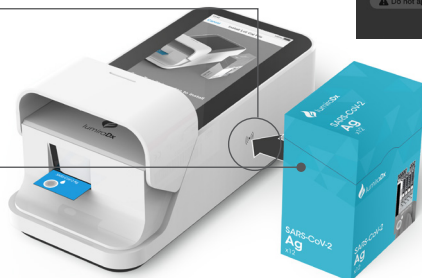

Installation

Place and hold back of Test Strip Carton ((Ⓢ)) symbol to install.

Confirmation

The Instrument will sound and a confirmation message will be displayed.



2.9 Extended Functionality and Operating Modes

The LumiraDx Instrument can be used in Standalone mode, or two Connected modes: "Managed" and "EHR Connected" (meaning connected to the organization's electronic health record). Refer to the LumiraDx Connect User Manual for further information.

Standalone

Run patient tests and quality control tests.

Store test results together with time, date and patient ID.



LumiraDx Instrument

Managed

Configure single or multiple Instruments.

Create WorkGroups and user login credentials.

Manage compliance and governance functions.



**LumiraDx Instrument
& LumiraDx Connect Hub
or LumiraDx Connect App**



**LumiraDx Connect
Manager**

EHR Connected

Integrate with Electronic Health Record for transfer of patient test results.

View patient results in your LIS or HIS.



**LumiraDx Instrument
& LumiraDx Connect Hub
or LumiraDx Connect App**



**LumiraDx Connect
Manager**




**Electronic Health
Record**

3 Preparation for Testing

All samples are potentially infectious and are capable of transmitting pathogens between patients and healthcare professionals.

3.1 Handling Test Strips

When you are ready to perform a test, select a Test Strip and check the expiry date. Discard the Test Strips if they are beyond the expiration date. If within the expiry date open the Test Strip Carton, take out one Test Strip, and remove it from the foil pouch from the end indicated by the tear triangles. Refer to the LumiraDx Test Strip Product Insert for further information on handling and stability.

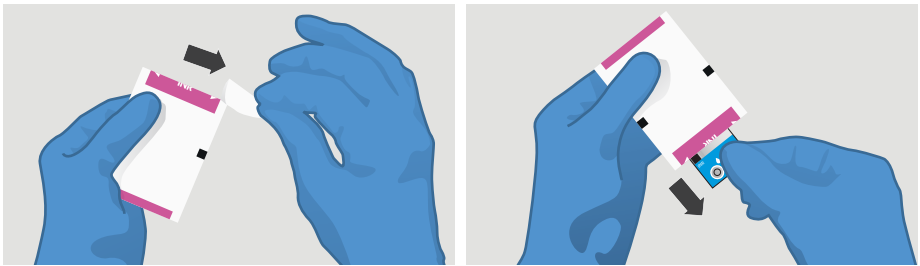

3.2 Inserting and Removing Test Strips

Hold the Test Strip by gripping the blue label end with the label facing upward.

Line up the black alignment rib on the Test Strip with the black line on the Instrument. Insert the Test Strip gently as far as it will go. The Instrument will sound when the Test Strip is detected if sounds are turned on in the Instrument settings (see section 4.5).

To remove a Test Strip from the Instrument, gently pull the blue label and pull straight out.

Do NOT touch Test Strip Sample Application Area.

Do NOT bend or fold the Test Strip.

Do NOT touch Test Strip contacts.





Test Strip contacts


Alignment rib

Line up black line on Test Strip with black line on Instrument.

Test type

Sample Application Area

Apply sample or Quality Control solution **AFTER** inserting Test Strip.

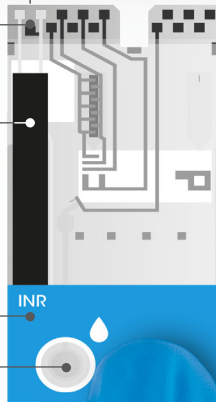

Note: Test Strip may appear differently.

3.3 Sample Collection and Application


When collecting any type of sample, follow universal collection precautions and guidelines according to your organization. Refer to Test Strip Product Insert for information on sample collection and processing.

For collection of capillary fingerstick samples: Before lancing ask the patient to rinse their hands in warm water and dry them. This is good practice to remove any residue and to increase the blood flow in the finger.

Details of LumiraDx recommended transfer tubes and swabs are available at lumiradx.com.

3.4 Applying a Sample to the Test Strip

Apply one drop of the sample to the Test Strip. Ensure that the sample is applied centrally and completely fills the sample application area in one clean drop.

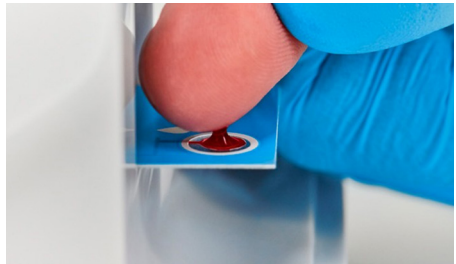

When the sample is detected the Instrument will sound and a confirmation message will be displayed.



The touchscreen of the LumiraDx Instrument will request the user to close the door immediately. Sample detection can take up to 15 seconds.

Do NOT apply more than one drop of sample.

Do NOT touch the Test Strip until the test has finished and the result is displayed.



3.5 Quality Control Solution Application

Refer to LumiraDx Quality Control Product Inserts for information on testing with LumiraDx Quality Controls.

4 Instrument Operation


The LumiraDx Instrument can be used in three modes; Standalone, Managed or EHR Connected.

This chapter describes how to perform first time setup, set preferences, perform patient and Quality Control tests, and view result history. Extended functionality for Connected Instruments is also described.

Please note:

- Instructions are combined with example screenshots. Some screens may appear differently depending on the test or mode of operation. All screens, test names and results displayed in this Platform User Manual are intended only as examples.
- You can only hear the sounds on the LumiraDx Instrument when they are turned on. This Platform User Manual presumes that the sounds are turned on.

4.1 Home Screen

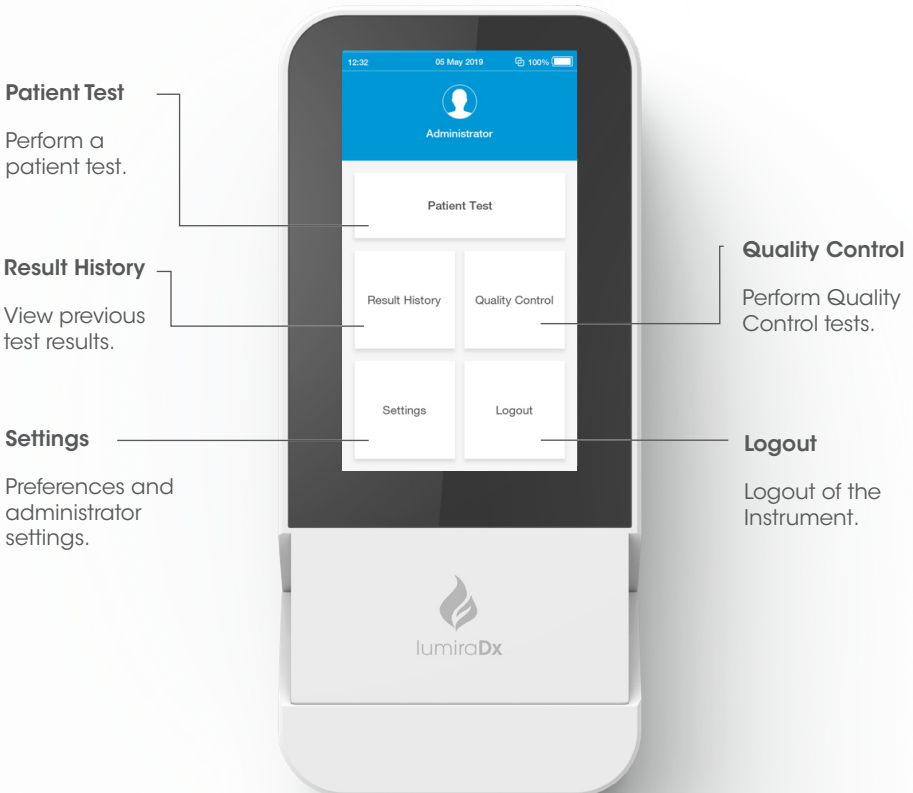

4.2 Settings Menu

4.2.1 General User Settings Menu

All users can set display and sound preferences in the Settings menu.

Display

Sets the brightness of the display.

Sounds

Key clicks and audible alerts can be turned on or off.


NOTE: For further information on display and sounds refer to section 4.5.

About

The About screen provides information on Instrument software and hardware versions, and serial number. For Connected Instruments the screen will also display the Organization, Instrument and WorkGroup names.

Legal

Gives details about licenses, Privacy Statement and Warranty.

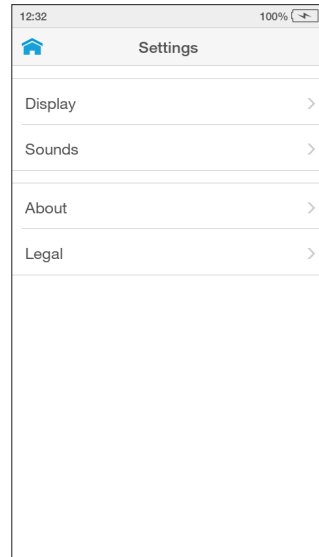

4.2.2 Administrator Settings Menu

An Administrator has additional options for setting preferences and settings.

Language

Sets the language of the text appearing on the display. The language is set to English by default. A different language can be selected from the list.

Date and Time

Sets the date and time of the Instrument. The time can be set in a 24-Hour format if required.

IMPORTANT: It is important that this is accurate to ensure the correct date and time is on the Test Result.

Automatic Power Off

Sets an automatic power off time between 1 and 18 hours (default is 18 hours).

Test Management

Allows an Administrator to:

- Pre 'Install Lot Calibration File'.
- Set the 'Assay Units'. Selects units of measure for Test Results. Select the Test from the list.

One or more Result Unit Formats can be selected by using the sliders that sets one or multiple result units for that Test.


- Set the 'Number Format', to set the decimal separator for results to either "." or ",", according to local preferences. For instance, a result can be represented as either "9.8 mg/L" or "9,8 mg/L".
- The 'Pooled Test' slider enables Pooled Testing on the Instrument.

NOTE: For further information on Test Management refer to section 4.6.

Factory Reset

Deletes all data currently stored on the Instrument, including result history, settings and Lot Calibration Files.

IMPORTANT: Factory reset should only be used under the instruction of LumiraDx.

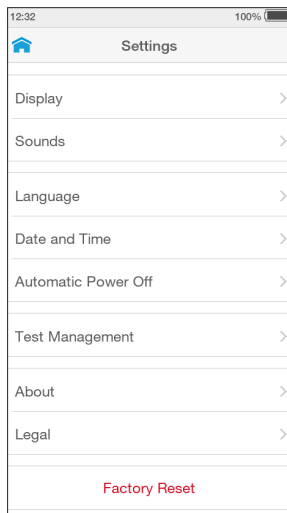

4.3 Standalone Instrument First Time Setup

For Standalone mode Instruments the language and current date and time will need to be set by the administrator prior to performing a patient test. Other sound and display preferences can be set by any user.

Administrator and general user passwords for Standalone operation are provided on a tear out sheet in the Platform Quick Reference Guide.

1. Power On


Power on the Instrument by pressing the power button at the rear of the Instrument. You will hear the Instrument powering on, and the display will be a blank black screen for several seconds before starting up. If the screen is just dimmed tap the touch-screen to wake up the Instrument.



2. Enter Password

Tap the password field to display the keyboard.

Enter password (using administrator password supplied) and tap 'Login' to complete the login.

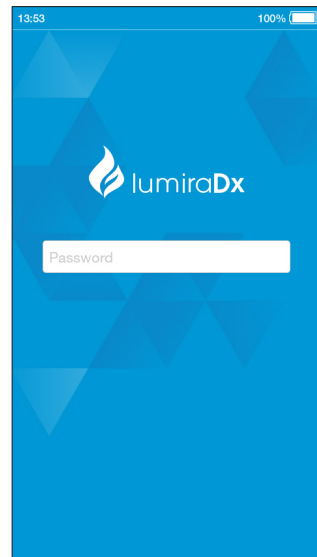


3. Set Language

The language is set to English by default. To set a different language go to 'Settings' and tap 'Language'.

Choose a language from the list for the text appearing on the display. The currently set language is ticked.

Tap 'Done' to save and return to the Settings menu.

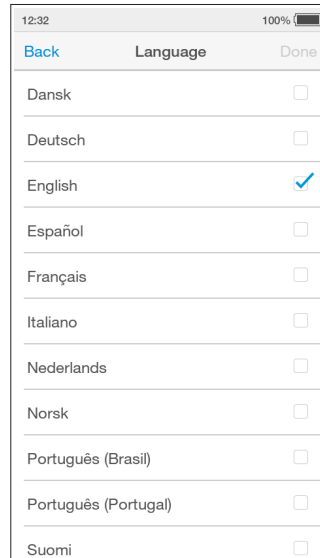

4. Set Date and Time


Go to 'Settings' and tap 'Date and Time'.

Tap the date field and use the selection wheel to set the current date. Tap 'Enter'.

Tap the time field and use the selection wheel to set the current time. Tap 'Enter'.

Tap 'Done', then check the date and time are correct and tap 'Confirm' to save.

Return to the home screen and tap 'Logout'.

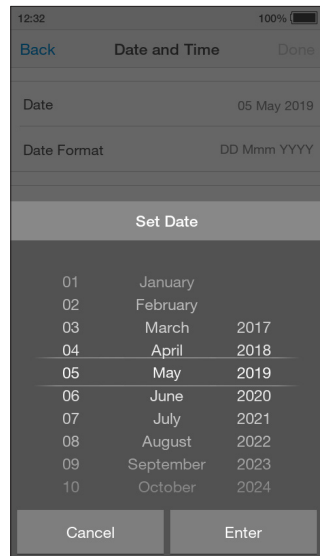


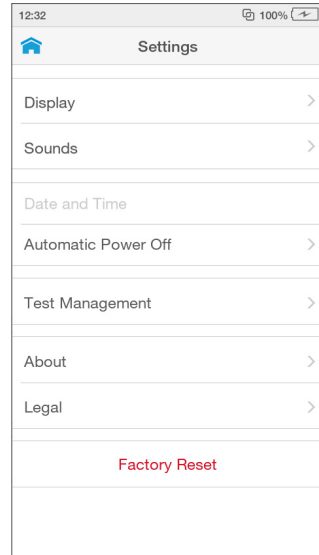
4.4 Connected Instrument First Time Setup

Connected system setup should be completed by the administrator on Connect Manager. Refer to the LumiraDx Connect User Manual for further information.

Power on by pressing the power button at the rear of the Instrument. You will hear the Instrument powering on and the display will be blank for several seconds before starting up. Follow the steps for Standalone first-time setup to set the language.

To download other Instrument settings, ensure the Instrument is within range of a LumiraDx Connect Hub or a device running the LumiraDx Connect App (within approximately 10 metres).

If  appears at the top of the screen, the Instrument is connected to Connect Manager and the Instrument settings will download and install automatically. Time, date, and language settings will be set automatically if the Instrument is in range of a LumiraDx Connect Hub and these settings will be greyed out in the settings menu.



Downloading settings for first time setup will take up to 4 minutes.

A ☁ will appear at the top of the screen for EHR Connected Instruments. This integrates the LumiraDx Instrument with the existing Electronic Health Record for transfer of patient test results. Patient test results can then be viewed within the LIS or HIS.

The Instrument status bar displays a 📶 when the Instrument is connected to Connect Manager



The Instrument status bar displays a ☁ when the Instrument is connected to the Electronic Health Record

4.5 Display and Sound Preferences


All users can set display and sound preferences in the Settings menu.

Display

In the Display menu the brightness of the display can be set.

Go to 'Settings' and tap 'Display'. Use the slider to adjust the brightness.

Tap 'Done' to save and return to the Settings menu.

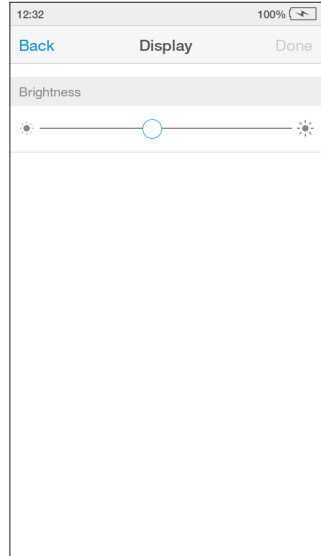


Sounds

In the Sounds menu the key clicks and audible alerts can be turned on or off.

Go to 'Settings' and tap 'Sounds'. Set sounds on or off, or to set the volume using the slider.


Tap 'Done' to save and return to the Settings menu.

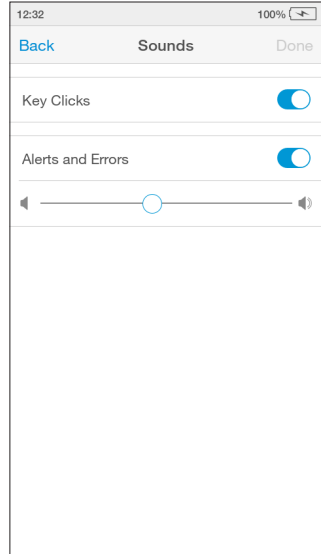

4.6 Test Management

Only an Administration User can use Test Management in the Settings menu.

Go to 'Settings' and tap 'Test Management'.

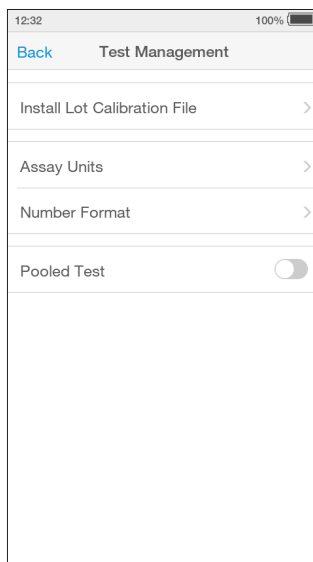

Install Lot Calibration File

A Lot Calibration File can be pre installed so they are ready to be used.

Go to 'Settings' and tap 'Install Lot Calibration File'.

Locate the ((•)) symbol on the Instrument.

When instructed by the Instrument, place and hold the Test Strip Carton against the RFID ((•)) icon to install.

The Instrument will sound (if sounds are turned on) and a confirmation message will be displayed once installed.



Assay Units

Units of measure can be configured to display on Tests Results.

NOTE: For some Test Results there may not be an option to configure units of measure as this varies by test type.

NOTE: The list of Tests available will depend what is available in your region.

Go to 'Settings' and tap 'Assay Units'.

Select the Test from the list.



Using the sliders select the units displayed on the Test Result.

NOTE: When the slider is moved to the right, highlights a units of measure is turned on.

NOTE: The maximum selectable units for a Test may vary by test type.

Tap 'Done' to save and return to the Settings menu.

IMPORTANT: Changing units will change the displayed and saved units of the measure for a Test. Refer to your organizational policies before changing.

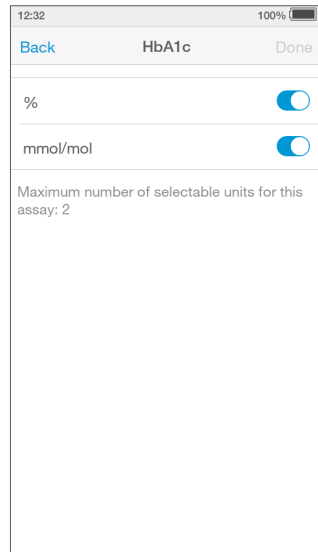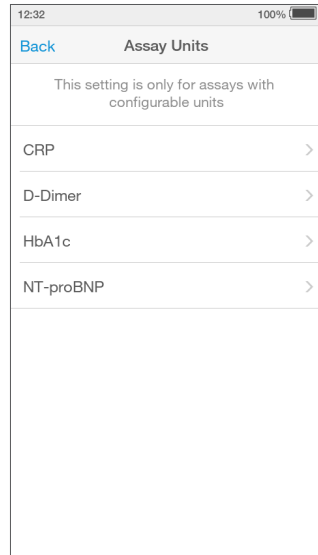

Number Format

Set the 'Number Format', to set the decimal separator for results to either "." or "," according to local preferences. For instance, a result can be represented as either "9.8 mg/L" or "9,8 mg/L".

Go to 'Settings' and tap 'Number Format'.



Select an option from the list by adding a tick in the box against "100.00" or "100,00".

Review the option selected and 'Confirm'.

Tap 'Done' to save and return to the Settings menu.

Note: Changing 'Result Unit Format' and 'Number Format' does not change existing results that are already stored in the Instrument and/or in Connect or other platforms.

IMPORTANT: Changing Number Format will change the displayed and saved Number Formats on all results. Refer to your organizational policies before changing.

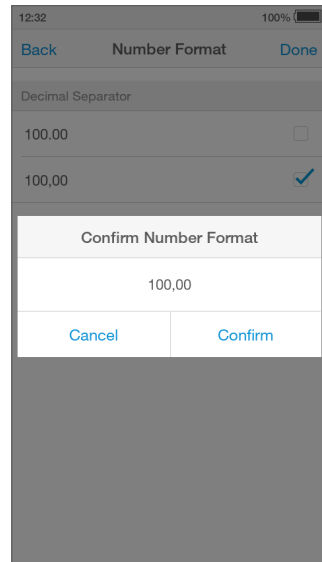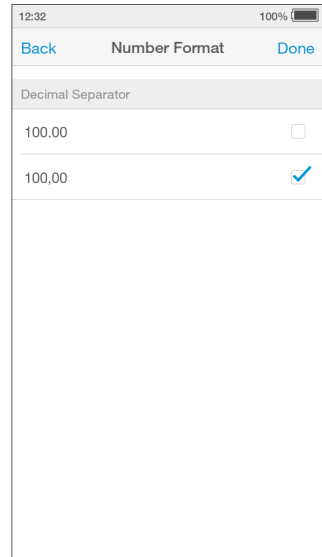


Pooled Test

The 'Pooled Test' slider enables Pooled Testing on the Instrument.

Go to 'Settings' and use the slider to select 'Pooled Test'.

When the slider is moved to the right, highlights this functionality is turned on.

NOTE: Pooled Test may not be available in some countries.

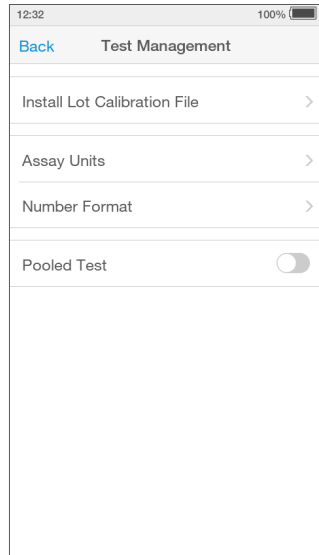

4.7 Performing a Patient Test

This Platform User Manual describes the basic patient test method for the LumiraDx Instrument. If there are differences in the test method for a particular diagnostic test, the Instrument will prompt at each step of the process. Please refer to the LumiraDx Test Strip Product Insert before performing a new test for specific test information.


Important Notes

ALWAYS

- Place the Instrument on a level, stable surface before starting a test.
- Ensure the Instrument is powered on before inserting a Test Strip.
- Apply sample **AFTER** inserting the Test Strip and the Instrument prompt.
- Follow the test operating conditions in the LumiraDx Test Strip Product Insert.
- Follow the information on correct handling of Test Strips in the LumiraDx Test Strip Product Insert.



To view test information during a patient test, or on the test result screen, tap the  at the top of the touch-screen.

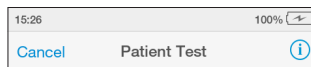

To cancel a patient test tap 'Cancel' at the top of the touch-screen and follow the Instrument prompts.

1. Power On and Login

Power on the Instrument by pressing the power button at the rear of the Instrument. You will hear the Instrument powering on and the display will be a blank black screen for several seconds before starting up. If the display is dimmed tap the touch-screen to wake up the Instrument.

If using a Standalone Instrument with just a password field, tap the password field to display the keyboard.

Enter password and tap 'Login' to complete the login.



If using a Connected Instrument, tap the User ID field to use the keyboard to enter User ID. Alternatively, scan User ID using the Barcode Scanner. Refer to the "Ancillary devices" chapter in this Platform User Manual for scanning instructions.

Tap the password field to display the keyboard.

Enter password and tap 'Login' to complete the login.




2. Check Date and Time

Check that the date and time are correct. If not correct, the current date and time will need to be set by the administrator. Refer to "First time setup" in this chapter for more information.

3. Tap Patient Test

Tap the 'Patient Test' button on the home screen.

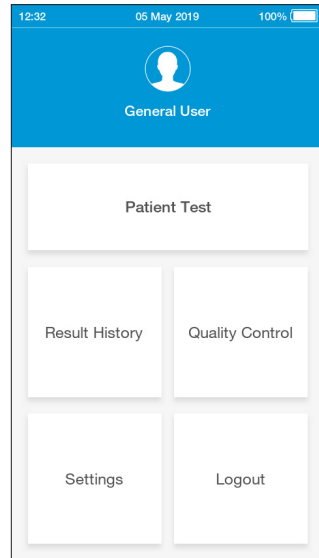


4. Enter Patient Details

Tap any field to use the keyboard to enter patient details, or scan the patient ID using the Barcode Scanner. Refer to the "Ancillary devices" chapter in this Platform User Manual for scanning instructions.

Use the 'Enter' button on the keyboard to move from field to field, or tap a field to enter additional details.

Complete all available patient details, then tap 'Next' to continue.


Note: Tap 'Next' if no patient details are available.

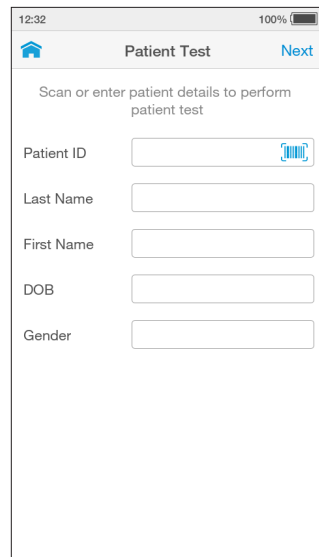

5. Using the Instrument in EHR Connected Mode

If the Instrument has an active connection to the EHR, scan the Patient ID using the Barcode Scanner or search for the patient in the HIS or LIS by entering patient details, then tap 'Search'.

Note: If no patient details are available, tap 'Search' without adding any details. To search for a patient either the full Patient ID or Last Name must be entered.

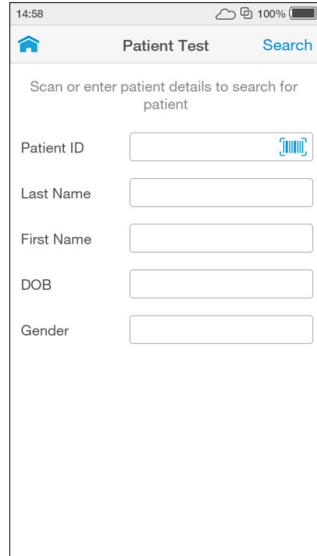

14:58 100%

Home Patient Test Search

Scan or enter patient details to search for patient

Patient ID

Last Name

First Name


DOB

Gender

EHR Connect Patient Search



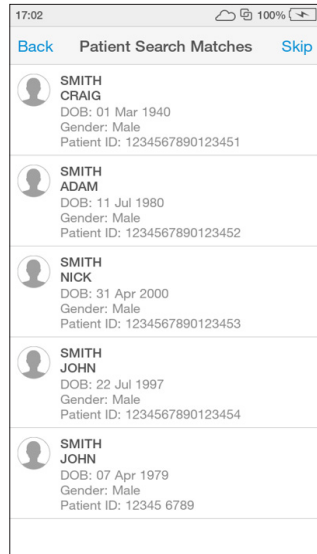


If only one patient matches the details entered, the Instrument will show the 'Confirm Patient Details' screen. If between 2 and 5 patients match the search details entered, the list of matches will be shown. Select the correct patient from the search list.

Note: Tap 'Skip' to proceed without selecting one of the patients available in the search list.




17:02 100%

Back Patient Search Matches Skip

	SMITH CRAIG DOB: 01 Mar 1940 Gender: Male Patient ID: 1234567890123451
	SMITH ADAM DOB: 11 Jul 1980 Gender: Male Patient ID: 1234567890123452
	SMITH NICK DOB: 31 Apr 2000 Gender: Male Patient ID: 1234567890123453
	SMITH JOHN DOB: 22 Jul 1997 Gender: Male Patient ID: 1234567890123454
	SMITH JOHN DOB: 07 Apr 1979 Gender: Male Patient ID: 12345 6789


No Patient Match

If the search times out, no patient matches are found, or more than 5 patients matches are found a message will display. Tap 'Proceed' to proceed with the test without verifying the patient details or tap 'Cancel' to go back and try a new search.

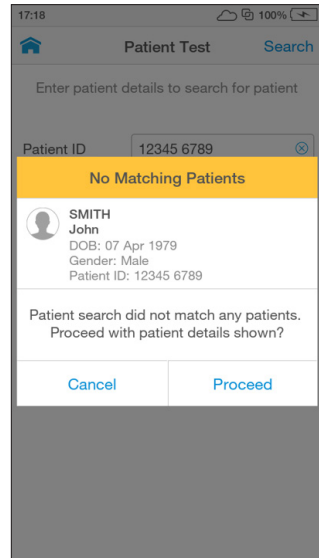

6. Confirm Patient Details

Check patient details are correct and tap 'Confirm' to proceed.

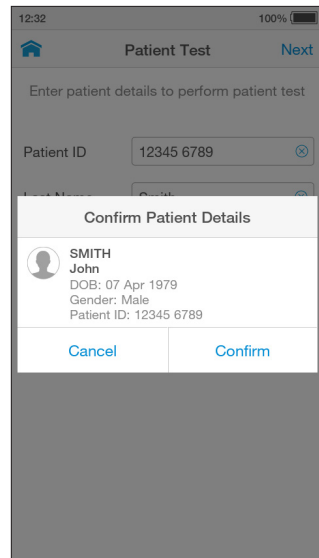

7. Open Door and Insert Test Strip


The Instrument display will prompt to open the door and insert a Test Strip.

Remove a Test Strip from its packaging and hold with the blue label side facing upward.

Line up the black alignment rib on the Test Strip with the black line on the Instrument. Insert the Test Strip gently as far as it will go.

The Instrument will sound when the Test Strip is detected.

- Do NOT** touch Test Strip Sample Application Area.
- Do NOT** touch Test Strip contacts.
- Do NOT** apply sample until prompted.

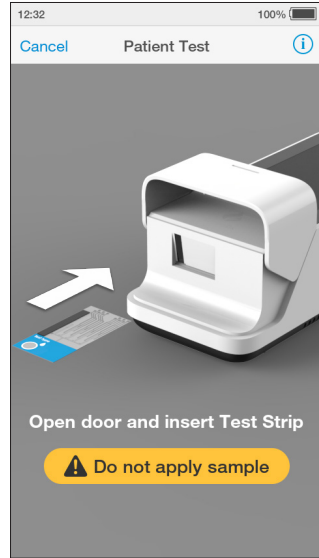


8. Select Sample Type

Select the intended sample type from the list displayed.

Sample type is test type dependent (refer to the LumiraDx Test Strip Product Insert for more information).

Note: The Instrument will prompt to install the Lot Calibration File when inserting a new Test Strip Lot.


Note: Capillary Blood refers to both finger stick and transfer tube applications.

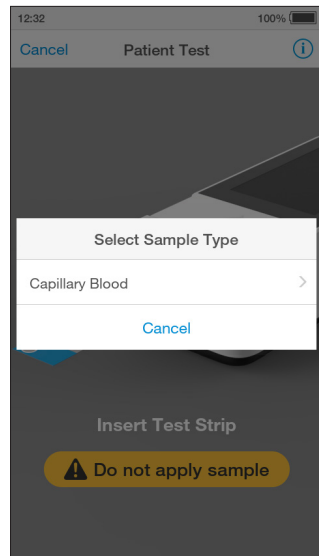

9. Confirm Test and Sample Type

Check the test and sample type is correct and tap 'Confirm' to proceed.

Note: A progress bar will indicate if the Instrument is heating the Test Strip.

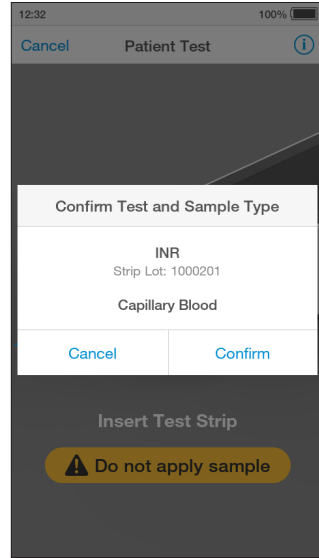

10. Apply Sample

In order to prepare the sample to be tested, please follow sample preparation steps in the Product Insert and Quick Reference Instructions.

The Instrument display will show a warning dialogue to remind the user to only apply one drop and the sample detection may take up to 15 seconds.


Apply one drop of the sample directly to the circular Sample Application Area on top of the Test Strip. The sample must cover the entire Sample Application Area. Refer to the "Handling Test Strips" chapter in this Platform User Manual for sample application instructions.

The Test Strip draws up the sample by capillary action.

When the sample is detected the Instrument will sound and a confirmation message will be displayed.

Note: Pay attention to the countdown for sample application to avoid test errors. This will be displayed as a countdown bar, and the Instrument will sound with 15 seconds remaining. If the sample is not applied by the end of the countdown the Instrument will display an error message.


Once the sample is applied it may take up to 15 seconds before the sample is detected.

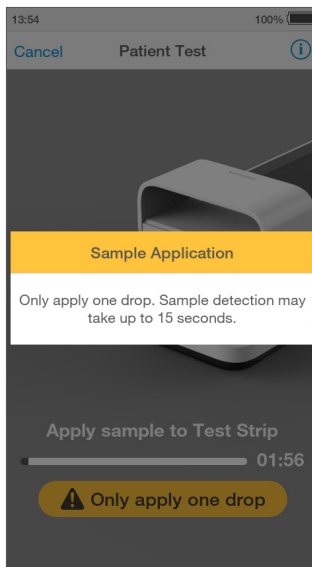

Do NOT apply more than one drop of sample.




Do NOT touch the Test Strip until the test has finished and the result is displayed.

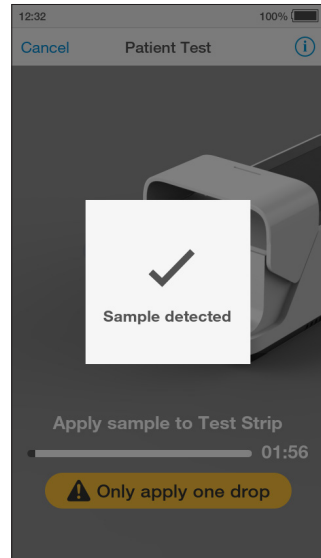

11. Immediately Close Door to Continue

The Instrument display will prompt to close the door.

Note: The countdown bar lasts for 10 seconds for the user to close the door. It is important to close the door quickly when prompted to avoid test errors. If the door is not closed by the end of the countdown the Instrument will sound and display an error message.

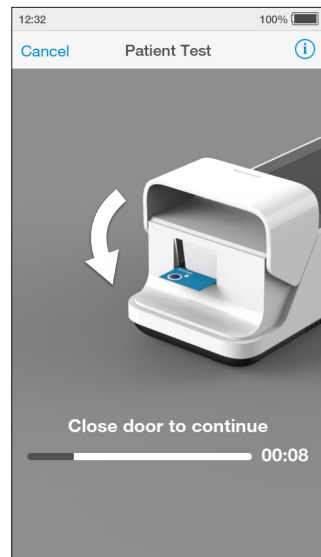


12. Test Processing

The test will start and a progress bar will be displayed whilst the test is processing.

Note: Each test type has a different processing time.

Do NOT open the door or move the Instrument during the test or an error will result and the test result will not be saved.

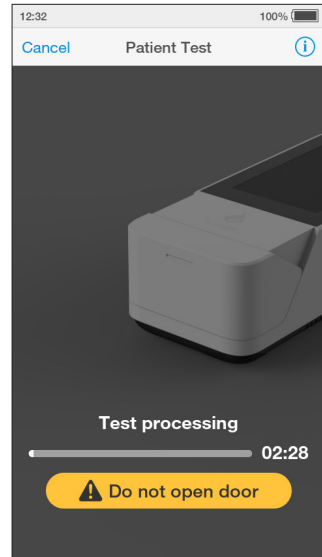

13. Review Result

When the test is complete the result will be displayed on the screen (an example of a quantitative result is shown here).

Results that are above or below the analytic range are displayed in red and indicated by the symbols > (above) or < (below).

If a LumiraDx Printer is connected to the Instrument, a 'Print' option will be available. Tap 'Print' to print the test result.


Review the result then tap 'Finish' or tap 'Comment' to add a comment, if required. A comment may be up to 50 characters in length.

To reject a test result, go to 'Comment' and tap 'Reject result'. A comment must be added if a test result is rejected.

Once the desired comment(s) are entered tap 'Done' to return the result screen.

Tap 'Finish' to save the test result to Result History and complete the test.

Note: If operating the Instrument in a connected mode, you must tap 'Finish' for the result and any comments to be sent to the EHR.

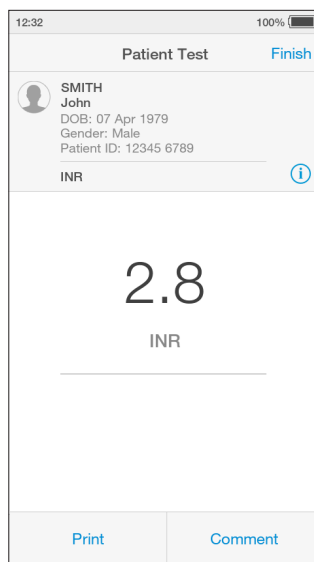

14. Remove and Dispose of Test Strip

The Instrument will prompt to remove the Test Strip.


Gently pull the blue label end of the Test Strip to remove and dispose.

IMPORTANT: Keep the test strip horizontal when removing from the Instrument.

Disinfect the Instrument at least once per day when in use or if contamination is suspected, unless recommended otherwise for specific tests. Refer to the "Cleaning and Disinfecting" chapter in this Platform User Manual for further information.

Close the Instrument door.

Dispose of all Test Strips used for patient testing safely in accordance with local regulations and procedures.

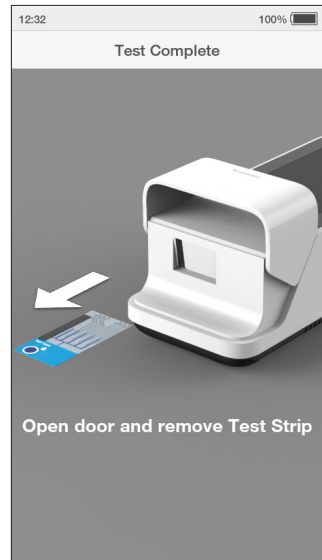

4.8 Quality Control

The LumiraDx Instrument has a number of built-in QC functions. In addition to these, LumiraDx Quality Controls can be used to meet compliance requirements as required.

To perform QC testing with LumiraDx Quality Controls, you will need:

- LumiraDx Instrument
- LumiraDx Test Strips
- LumiraDx Quality Controls

Preparing for a QC test requires the same steps as preparing to perform a patient test. The only difference in the test method is the use of Quality Controls instead of patient samples.

Instruments being managed in a Connected mode may have the Quality Control policy set in the Connect Manager system to:

- **Off**
- **Mandatory** – If the Quality Control policy has expired users are unable to perform a patient test, until a Quality Control Test has been performed with a status of PASS.

- **Advisory** – If the Quality Control policy has expired a warning is displayed on the Instrument. Users can acknowledge the warning and proceed with testing.
- **Mandatory with Override** – If the Quality Control policy has expired, users are required to either run a Quality Control Test or override the expired Quality Control policy and run a patient test by entering a comment to explain the reason for the override.

The timing and frequency of mandatory Quality Control tests are configurable in Connect Manager by Policy:

- **Lot** – Users are required to perform a Quality Control test when using a new Test Lot on an Instrument for the first time.
- **Type** – Users are required to perform a Quality Control test when using a new Test Type on an Instrument for the first time.

For information on LumiraDx Quality Controls refer to the LumiraDx Quality Control Product Inserts.



4.9 Performing a Quality Control Test


This Platform User Manual describes the basic Quality Control test method for the LumiraDx platform. If there are differences in the test method for a particular QC test, the Instrument will prompt at each step of the process. Please refer to the LumiraDx Test Strip Product Insert before performing a new QC test for specific information.


Important Notes

ALWAYS

- Wear a clean pair of gloves before performing a QC test.
- Place the Instrument on a level, stable surface before performing a test.
- Use only LumiraDx Quality Controls.
- Apply Quality Control **AFTER** inserting the Test Strip and the Instrument prompt.
- Follow the test operating conditions in the LumiraDx Test Strip Product Insert.
- Follow the information on correct handling of Test Strips in the LumiraDx Test Strip Product Insert.



To view test information during a QC test, or on the test result screen, tap the  at the top of the touch-screen.

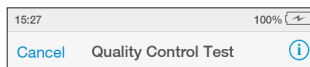

To cancel a QC test tap 'Cancel' at the top of the touch-screen and follow the Instrument prompts.

1. Power On and Login

2. Check Date and Time


Check that the date and time are correct. If not correct, the current date and time will need to be set by the administrator. (Refer to "First time setup" in this chapter for more information).

3. Prepare Quality Control Solution

Prepare the Quality Control according to the LumiraDx Quality Control Product Insert.


4. Tap Quality Control

Tap the 'Quality Control' button on the home screen.

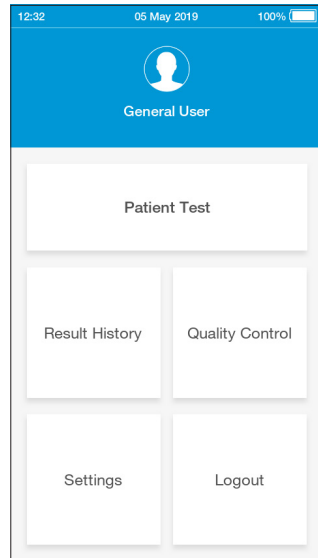

5. Select Quality Control Test

Tap 'Quality Control Test' from the Quality Control menu.

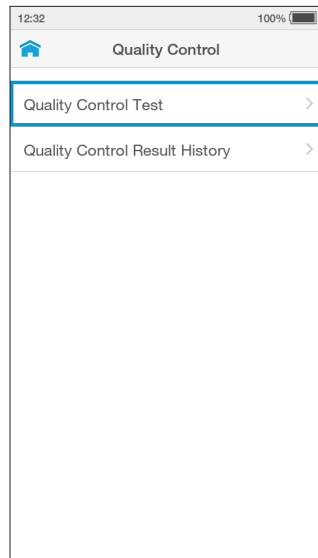

6. Open Door and Insert Test Strip

The Instrument display will prompt to open the door and insert a Test Strip.

Remove a Test Strip from its packaging and hold with the blue label side facing upward.


Line up the black alignment rib on the Test Strip with the black line on the Instrument. Insert the Test Strip gently as far as it will go.

The Instrument will sound when the Test Strip is detected.

Do NOT touch Test Strip Sample Application Area.

Do NOT touch Test Strip contacts.


Do NOT apply Quality Control until prompted.

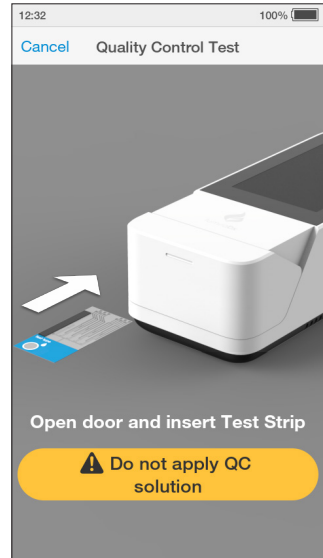

7. Confirm test type

Check the test type is correct and tap 'Confirm' to proceed.

Note: The Instrument will prompt to install the Lot Calibration File when inserting a new Test Strip Lot.

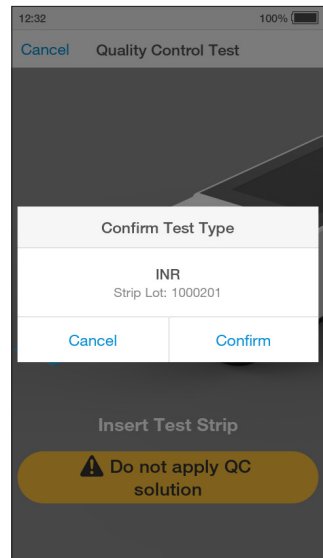


8. Select Quality Control Level

Select the intended Quality Control from the list displayed.

Please refer to the LumiraDx Quality Control Product Insert for more information.

Note: The Instrument will prompt when to apply the Quality Control solution to the Test Strip.

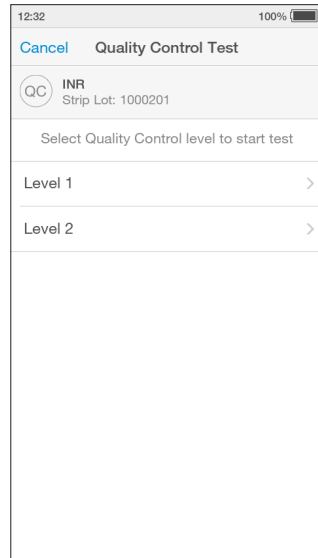


9. Enter Quality Control Lot Number

Tap the input field to use the keyboard to enter the 16 digit Quality Control Lot number, and tap 'Next' to continue.

Alternatively scan the Quality Control barcode using the Barcode Scanner. Refer to the "Ancillary devices" chapter in this Platform User Manual for scanning instructions.

Note: If the barcode from a different QC level or test from the one selected on the previous screen is entered an error message will display.

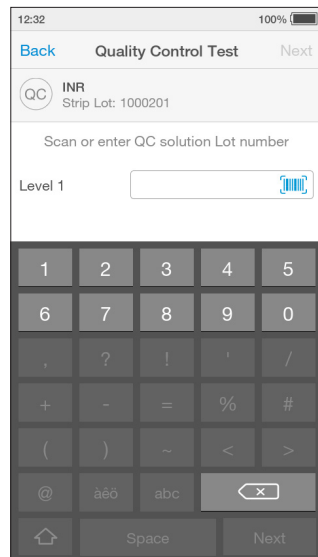

10. Apply Quality Control Solution

The Instrument display will prompt to apply the Quality Control sample.

The Instrument display will show a warning dialogue to remind the user to only apply one drop and the sample detection of the Quality Control Solution may take up to 15 seconds.

Apply the Quality Control sample directly to the circular Sample Application Area on top of the Test Strip. The solution must cover the entire application area. Refer to LumiraDx Quality Control Product Inserts for additional information on testing with LumiraDx Quality Controls.

Note: Pay attention to the countdown for sample application to avoid errors. If the Quality Control Solution is not applied by the end of the countdown the Instrument will display an error message.

When the Quality Control sample is detected the Instrument will sound and a confirmation message will be displayed.

Once the sample is applied it may take up to 15 seconds before the sample is detected.



Do NOT apply more than one drop of Quality Control sample.



Do NOT touch the Test Strip until the test has finished and the result is displayed.



11. Immediately Close Door to Continue

The Instrument display will display a prompt to close the door.

The test will start and a progress bar will be displayed whilst the test is processing.

Note: Pay attention to the countdown for closing the door to avoid test errors. This will be displayed as a countdown bar for only a few seconds. If the door is not closed by the end of the countdown the Instrument will sound and display an error message.




12. QC Test Processing

The QC test will start and a progress bar will be displayed whilst the test is processing.

Note: Each test type has a different processing time.

Do NOT open the door or move the Instrument during the test or an error will result and the test result will not be saved.

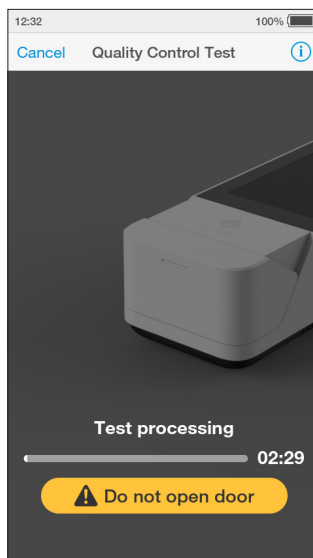

13. Review QC Result


When the QC test is complete the result, the acceptable range of results and whether the QC test has passed or failed will be displayed on the screen. An example result is shown here.

Review the result then tap 'Finish' or tap 'Comment' to add a comment, if required. A comment may be up to 50 characters in length.

Once the desired comment(s) are entered tap 'Done' to return the result screen.

If a LumiraDx Printer is connected to the Instrument, a 'Print' option will be available. Tap 'Print' to print the test result.

Tap 'Finish' to save the test result to Result History and complete the test.

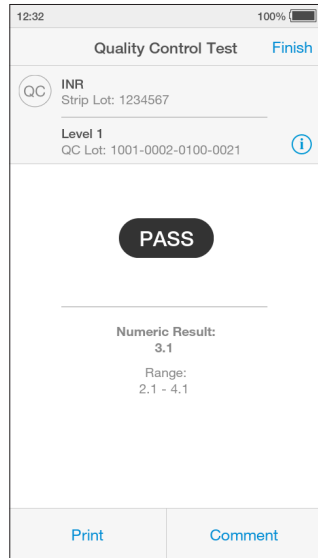

The range of results acceptable for each QC level is also detailed in the LumiraDx Quality Control Ranges Product Insert in the Test Strip Carton.



14. Remove and Dispose of Test Strip


The Instrument will prompt to remove the Test Strip.

Gently pull the blue label end of the Test Strip to remove and dispose.

IMPORTANT: Keep the test strip horizontal when removing from the Instrument.

Close the Instrument door.

Dispose of all Test Strips used for Quality Control testing safely in accordance with local regulations and procedures.

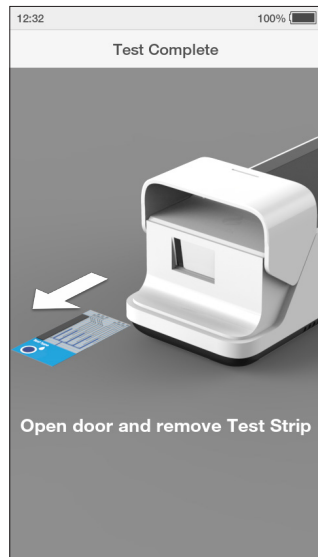


15. Complete Remaining QC Levels

Multiple QC levels may be required depending on test type.


Refer to the LumiraDx Quality Control Product Insert for more information on QC levels.

Repeat steps in "Performing a Quality Control test" to test any additional QC levels.

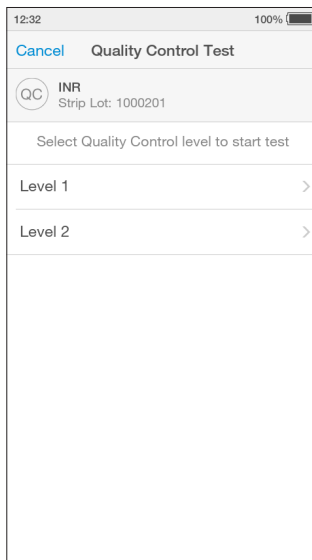


16. Mandatory Quality Control Policy

Connected Instruments with a Quality Control policy enabled will have an additional menu item of 'Quality Control Status'.

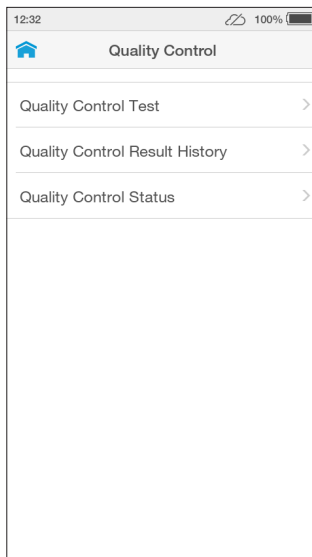


Quality Control test statuses are sorted by 'Recent Tests' meaning test Lots for which patient tests or Quality Control tests have been performed recently. 'Other Tests' are test Lots which have not had a patient test or QC test performed for more than 90 days.

Note: QC statuses can also be filtered by test type.

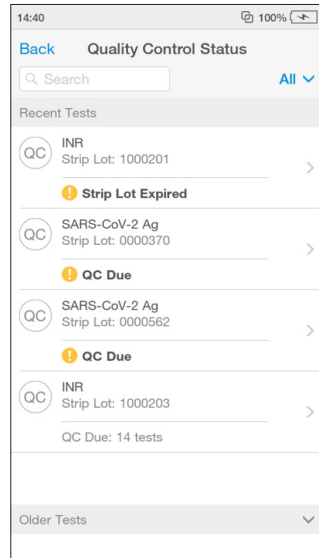


The Quality Control level selection screen will also show the status of each QC level as well as the date of any recently performed QC tests.

After completing a QC test level, the Instrument will return to the QC level selection page until all QC levels have been completed.

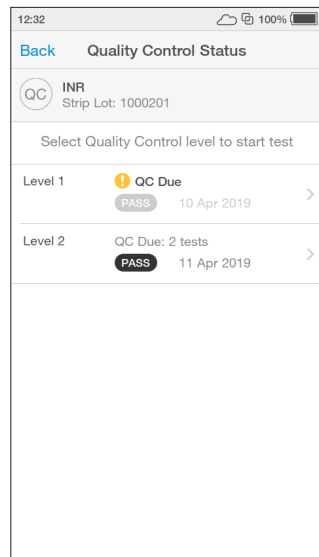

If a user attempts to perform a patient test using a Lot for which a mandatory Quality Control test is due, the Instrument will display an alert message and prompt the user to perform a Quality Control test.

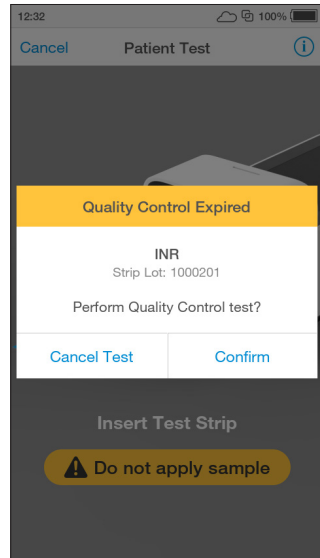


4.10 Result History

The LumiraDx Instrument can save 1000 patient or QC test results, together with respective time, date and comments. If the Instrument memory is full, the oldest result is automatically deleted.

1. Power On and Login

2. Tap Result History

To view Patient or QC test results, tap 'Result History'.

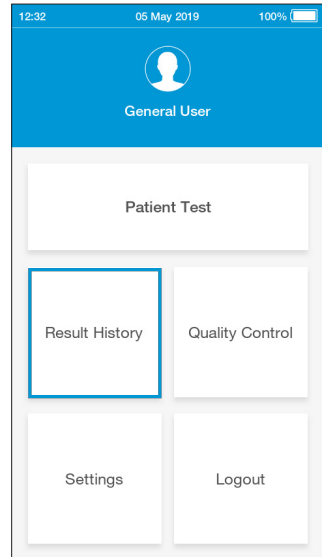

3. Review Test Results


All entries are shown in date order, with the newest at the top. Test results can be filtered by selecting Patient Tests, QC Tests, Rejected Tests or Invalid Tests.

Test results are marked as Invalid if a system error occurs during test processing. Refer to the "Troubleshooting" chapter of this Platform User Manual for information on errors.


Scroll through the list and tap any entry to view the test result in full.

Search for a patient test result by entering the Patient ID, First Name or Last Name in the search bar from either the Patient Tests, Rejected Tests or Invalid Tests screen.


Note: A search for a partial Patient ID, First Name or Last Name will only work for the start of the search term.

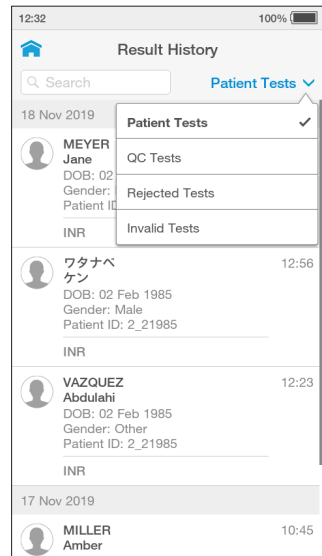


Any patient or QC test result can be printed from the test result details screen by tapping 'Print' when a LumiraDx Printer is connected to an Instrument USB port.

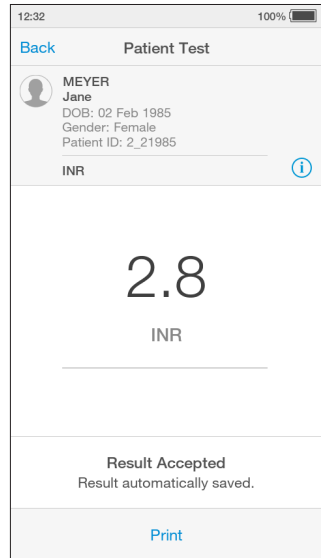

EHR Connected Instruments will indicate whether a test result has been sent to the electronic health record.

A  symbol on the home screen indicates that one or more test results have not yet been sent to the EHR.

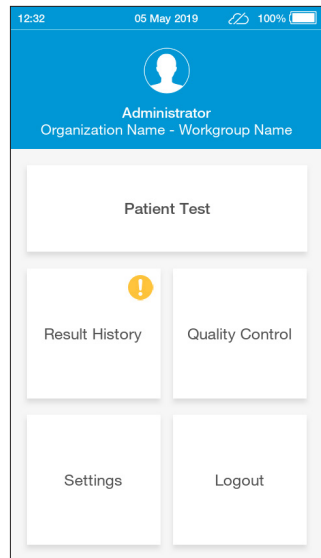


The Result History list will indicate whether test results have been sent to the EHR or not with 'Sent' and 'Not Sent'.

12:32		100%	
Result History			
Q Search		Patient Tests	
10 Apr 2019			
	NICOLE LARSEN DOB: 15 Sep 1978 Gender: Female Patient ID: 4_21834	14:35	
	INR	Not Sent	
	JANE MEYER DOB: 02 Feb 1985 Gender: Female Patient ID: 2_21985	14:02	
	INR	Not Sent	
	UNKNOWN DOE DOB: 23 Dec 1945 Gender: Male Patient ID: 23_121945	13:12	
	INR	Sent	
	JOHN SMITH DOB: 04 Oct 1989 Gender: Male Patient ID: 2_48573	12:38	
	INR	Not Sent	


4.11 Onboard Quality Control

In addition to the system checks performed by the Instrument during start up and within the test procedure, some LumiraDx Tests also run an onboard quality control assay alongside the patient test.


When a test has completed and the OBC test has passed, the test result will be displayed as normal. Tapping on the **i** on the result screen at the end of the test or from the Result History view shows additional details about the test, including **'Control: Valid'** which confirms that the OBC has passed.

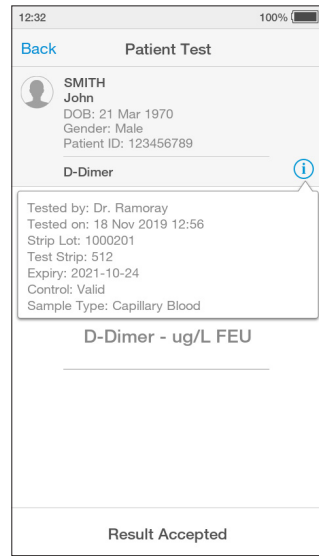

If the OBC fails, then an error message will be displayed and no test result will be returned. Follow the on screen instructions to dispose of the Test Strip and start a new test. If the problem persists, contact Customer Services.

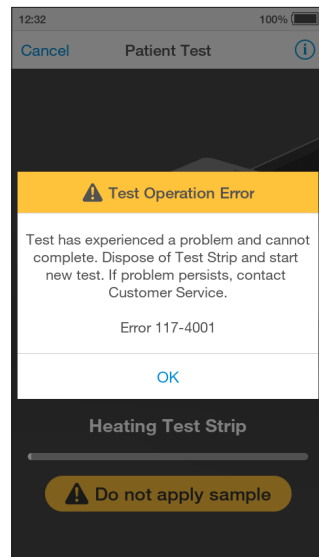

A test which fails the OBC will appear in the 'Invalid' filter of the Results History.

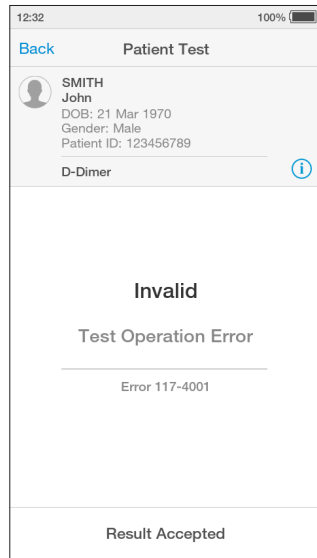

5 Cleaning and Disinfecting

It is important to observe the disinfection guidelines of your organization. Below are procedures for cleaning and disinfecting the Instrument. Failure to follow these procedures may cause Instrument malfunction.

The difference between cleaning and disinfecting:

Cleaning is the physical removal of dirt or other foreign material from the Instrument surface.

Disinfecting is the chemical removal of harmful microorganisms (pathogens) from the Instrument.

When should the Instrument be cleaned and disinfected?

The Instrument should be cleaned whenever it is visibly dirty. It is recommended to disinfect the Instrument with LumiraDx recommended disinfecting material at least once per day when in use or if contamination is suspected, unless recommended otherwise for specific tests. For more information, please refer to the Product Insert for the Test you are running.

What should be cleaned and disinfected?

The following parts of the Instrument can be cleaned and/or disinfected:

- The area around the Test Strip slot
- The entire Instrument housing
- The Instrument touch-screen
- All surfaces of the Instrument door


Cleaning and disinfecting



- Always wear gloves whilst cleaning and disinfecting the Instrument.
- Always use LumiraDx recommended materials to disinfect the Instrument. Details of LumiraDx recommended materials can be found at lumiradx.com.
- **CAUTION: Excessive liquid may damage the Instrument.** It is important for the protection of the Instrument that exposure to excess moisture is prevented. All disinfection cloths and/or wipes should only be slightly damp, with any excess liquid being manually removed from the cloth or wipe before use.
- **CAUTION:** Liquid should not be placed in or near the vents on the bottom or back of the Instrument.
- **CAUTION:** Alcohol wipes alone are not sufficient to disinfect the Instrument for blood-based samples, due to the potential presence of bloodborne pathogens.
- **Do NOT** spray or pour solution directly onto the Instrument.
- Following cleaning and disinfection, and before performing a patient test, change gloves and wash hands.



Cleaning Instrument touch-screen

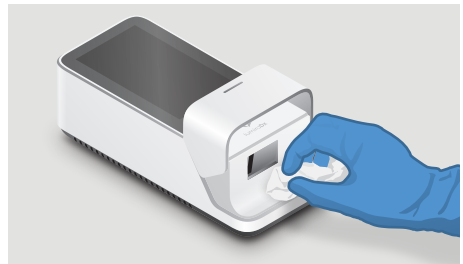

Cleaning the area around the Test Strip slot

5.1 Cleaning Procedure

Always wear gloves to clean the Instrument.

Wipe the external surfaces of the Instrument with a soft cloth that is slightly water dampened, but not wet.

CAUTION: Excessive liquid may damage the Instrument.

Dispose of cleaning materials in accordance with local biohazardous waste disposal procedures.

5.2 Disinfecting Procedure

5.2.1 General Guidance on Disinfecting the Instrument

Always wear gloves to disinfect the Instrument.

Before disinfecting the Instrument for the first time, it is necessary to remove the protective screen cover by peeling it off.

CAUTION: Excessive liquid may damage the Instrument. Prior to disinfecting, it is necessary to manually squeeze any excess liquid from disinfecting wipes or cloth. The wipe or cloth should be slightly damp, but not dripping or wet prior to disinfecting.

CAUTION: Liquid should not be placed in or near the vents on the bottom or back of the Instrument.

It is recommended to disinfect the Instrument with LumiraDx recommended disinfecting material at least once per day when in use or if contamination is suspected, unless recommended otherwise for specific tests. For more information, please refer to the Product Insert for the Test you are performing.

5.2.2 Disinfection Guidance – if only testing blood-based samples

It is recommended to disinfect the Instrument after each patient sample, or if contamination is suspected.

CAUTION: Excessive liquid may damage the Instrument. It is important for the protection of the Instrument that exposure to excess moisture is prevented. All disinfection cloths and/or wipes should only be slightly damp, with any excess liquid being manually removed from the cloth before use.

CAUTION: Alcohol wipes alone are not sufficient to disinfect the Instrument for blood-based samples, due to the potential presence of bloodborne pathogens.

Procedure:

- 1) Using a LumiraDx recommended disinfecting material, wipe the external surfaces of the Instrument while taking care to avoid the door hinges, Test Strip inlet, back and bottom vents, power cord, and USB port.
- 2) Allow the disinfectant at least **5 minutes** contact time before testing the next sample.
- 3) Dispose of disinfectant materials in accordance with local biohazardous waste disposal procedures

The Instrument is now ready to perform another test.

5.2.3 Disinfection guidance – if only testing swab-based samples

It is recommended to disinfect the Instrument at least once per day, or if contamination is suspected.

CAUTION: Excessive liquid may damage the Instrument. It is important for the protection of the Instrument that exposure to excess moisture is prevented. All disinfection cloths and/or wipes should only be slightly damp, with any excess liquid being manually removed from the cloth before use.

Procedure:

1) Using a LumiraDx recommended disinfecting material, wipe the external surfaces of the Instrument while taking care to avoid the door hinges, Test Strip inlet, back and bottom vents, power cord, and USB port.

2) Allow the disinfectant at least **1 minute** contact time before testing the next sample.

3) Dispose of disinfectant materials in accordance with local biohazardous waste disposal procedures.

The Instrument is now ready to perform another test.

For technical assistance or questions and information about LumiraDx approved materials go to **lumiradx.com**.

6 Software Updates

LumiraDx will contact administrators when software updates are available. Software updates will be provided on USB memory sticks and should be installed on each Instrument by the administrator. Installing the latest updates will help ensure that the LumiraDx Instrument operates with optimum performance and that the latest features are available. This section describes the steps required to complete the software update.

Note: Ensure that the Instrument is connected to a power supply before performing a software update.


IMPORTANT: Please save the latest USB Memory Stick in the event a re-installment of software update is required.

The USB Memory Stick is part of servicing activities related to the Instrument firmware and software. If the USB Memory Stick is lost or discarded, please contact Customer Services to receive a replacement.



6.1 Check Software Version

1. Check the software version currently installed on the Instrument

Login to the Instrument from the home-screen tap the settings button

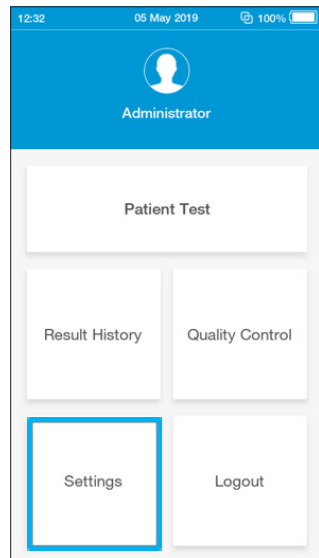


2. Tap the About button

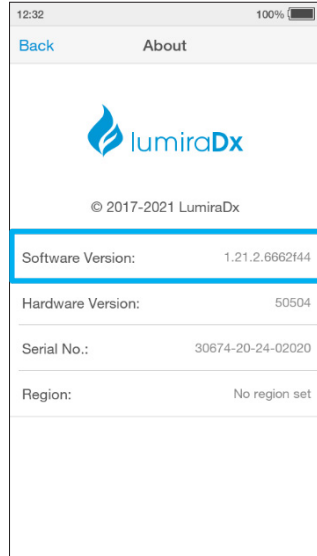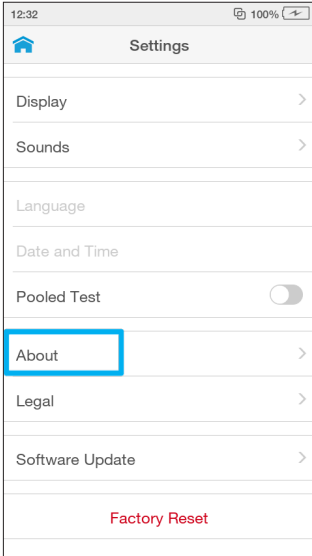

6.2 Performing a Software Update

1. Power OFF the Instrument
2. While the Instrument is powered OFF, Remove the USB Memory Stick from its packaging and insert it into the USB port nearest to the power button as shown in the image below.
3. Leave the USB Memory Stick inserted until after step 6.

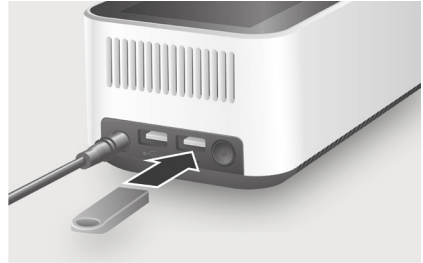


4. Power ON the Instrument

Power on the Instrument by pressing the power button at the rear of the Instrument.

A progress bar on the Instrument display will indicate installation.

Once installation is completed the Instrument will power on as usual.

5. Allow the Instrument to start up and complete the self-test.

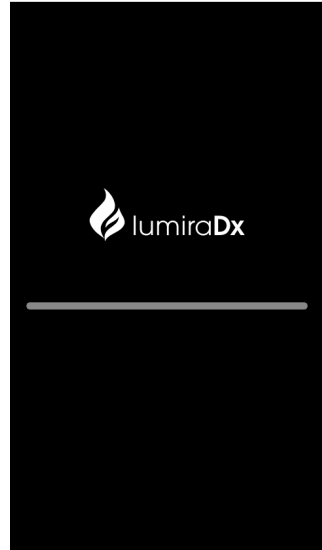

6. Once you reach the login screen, remove the USB stick.

IMPORTANT: Do not remove the USB Memory Stick before the login screen is shown.

IMPORTANT: Please save your USB Memory Stick in the event a reinstallation of software update is required.

IMPORTANT: If a factory reset has been performed on the Instrument after installing a software update, the software update must be reinstalled by following Steps 1-6.



7 Troubleshooting

The LumiraDx Instrument regularly performs internal checks for unexpected and unwanted conditions. These may arise for technical reasons or due to handling errors.

If an issue occurs, a message will be displayed on the Instrument touch-screen. Alert messages include useful information and are highlighted by an orange banner. Error messages also include a ▲ symbol. All messages will contain a description of the Instrument status or error and an instruction. Error messages contain an identifying code that may be used for further troubleshooting purposes.

Follow the instruction to resolve the issue. If the issue is resolved, continue using the Instrument as required.

If errors occur frequently or the error persists, please note the error code and contact Customer Services. If the Instrument has been dropped, do not proceed with use. In this case, also contact Customer Services for advice. Refer to the "Customer Services" chapter of this Platform User Manual for contact information.


An overview of the list of platform error messages can be found at **lumiradx.com/instrument**.

If a serious event has occurred, please contact LumiraDx Customer Services at **customerservices@lumiradx.com**.

Example alert message:

Description and Instruction

Tap 'OK' to proceed. Connect the power supply unit before the battery runs out.

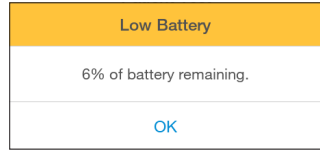


Example error messages: Error 002

Description and Instruction

The Instrument door has been opened while running a test so the test cannot be completed.

Dispose of Test Strip, start a new test and follow the instructions on the display.

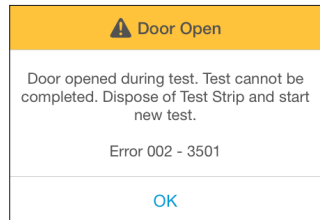

Errors and unusual behavior without error alerts

Some conditions may occur that do not result in an error or alert message. If the troubleshooting steps below do not resolve the issue or the problem reoccurs contact Customer Services.

Instrument does not power on

If pressing the power button does not power on the Instrument:


1. Connect the power supply unit. Wait 15 minutes and press the power button.

Cannot login

Do not have or know where to find the correct password:

1. Login with the password provided.
2. Tap the password field to bring up the keyboard and enter the password. Tap 'Login' to proceed.


Cannot connect Instrument

The Instrument is not displaying  symbol:

1. Check that the Instrument is within range of the LumiraDx Connect Hub or the device running the LumiraDx Connect App.
2. Ensure the LumiraDx Connect Hub is powered on and the green Connect light is on.

Lot Calibration File is not installed

Lot Calibration File is not installed:

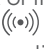1. Refer to the "Getting Started" chapter of this Platform User Manual for instructions on how to install the Lot Calibration File.
2. Ensure that the back of the Test Strip Carton is touching the  symbol on the side of the Instrument continually until the Instrument sounds and a confirmation message is displayed.

Instrument does not detect the Test Strip

If the Test Strip is not detected and no message is displayed on the touch-screen:

1. Check that the Test Strip is fully inserted.
2. Check for damage to the Test Strip. If damaged discard and insert a new Test Strip.

The Instrument will not start the test

If the Instrument will not start the test after applying the sample or Quality Control solution:

- Check the Instrument door is fully closed.

Incorrect date or time

If the date and time is incorrect:

- Contact the administrator for "Connected" Instruments, or refer to the "First time setup" section of this Platform User Manual for instructions on setting date and time.

Patient test result is red

If the patient test result is displayed in red:

- Test results that are above or below the analytic range are displayed in red and indicated by the symbols > (above) or < (below).

QC test failure

If the Quality Control test fails unexpectedly:

1. Check the Quality Control solution type and level matches the test type and QC range information.
2. Check the expiry date of the Quality Control solution.
3. Refer to the Quality Control Product Insert to ensure correct procedure is followed.

Retest using a new Test Strip. Ensure the Test Strip and Quality Control solution are handled carefully to prevent any contamination.

8 Maintenance and Disposal

The LumiraDx Instrument and ancillary devices do not require user maintenance and they have no user-serviceable parts. For user-performed software updates, see Section 6. No attempt should be made to open the Instrument or ancillary devices.

Changes or modifications not expressly approved by LumiraDx UK Ltd could void the user's authority to operate the Instrument.

In case of Instrument or ancillary devices failure or damage, or to arrange collection/disposal contact LumiraDx Customer Services.

9 Instrument Specifications


Operating temperature	15 °C to 30 °C (59 °F to 86 °F)
Storage temperature	-10 °C to 50 °C (14 °F to 122 °F)
Relative humidity	10 % to 90 % RH (non-condensing). Specific Relative Humidity conditions may be more restrictive please refer to the individual Product Insert for the specific test you are performing.
Maximum altitude	3,000 m (9,840 feet) operating
Pollution Degree	2
Data storage	1000 test results with date, time and comments Lot Calibration Files
Communications	2 x USB ports RFID Reader 13.56MHz, 0 dBi (EIRP) Bluetooth Low Energy 2.4GHz ISM band, 2400MHz to 2483.5 MHz, 0.5dBm (ERP)
Battery	Lithium Ion Polymer 7.4V 5000 mAh Approx. 20 tests per charge cycle
Dimensions	210 x 97 x 73 mm
Weight	1,100 g
Power supply unit	Manufacturer: CUI Model Number: SDM36-12-UD8-P94-C1 Overvoltage category: II Voltage fluctuations: +/- 10% Input 100-240V / 50-60 Hz / 1.0 – 0.5A Output: 12 V / 3 A
	Manufacturer: CUI Model Number: SMM36-12-K-P94-C1 Overvoltage category: II Voltage fluctuations: +/- 10% Input 100-240V / 50-60 Hz / 1.0 – 0.5A Output: 12 V / 3 A

10 Ancillary Devices

10.1 LumiraDx Barcode Scanner

The LumiraDx Barcode Scanner can be used to:

- 1) Capture patient or sample IDs.
- 2) Easily enter quality control Lot numbers.
- 2) Enter an operator username to log in to a LumiraDx Instrument. This can only be used if the Instrument is in connected mode. Please contact Customer Services to learn more.

 *Barcode symbol on screen indicates that you can scan for data entry. Tap symbol for Barcode Scanner instructions.*

Scanning distance

Hold scanner approximately 15 cm from barcode.

Scanner lights

Scan button

Press button to scan. A successful scan will be indicated by a beep sound and green light.



Insert USB connector into one of the USB ports at the rear of the Instrument.



LumiraDx Barcode Scanner Cradle



Attaching LumiraDx Barcode Scanner cradle to Instrument



Detaching LumiraDx Barcode Scanner cradle from Instrument



Note: Correct orientation of Barcode Scanner cradle

LumiraDx Desktop Stand



10.1.1 Getting Started

To get started, first check that the LumiraDx Barcode Scanner is already set up to use. On the Instrument Home Screen:

- Select "Patient Test"
- Scan the barcode below:



- Check that *69121847974* appears in the ID field.

If the above number does not appear or you get an error, follow the setup process below. If you use 2D or custom barcodes for sample tracking, contact Customer Services.

10.1.2 LumiraDx Barcode Scanner Basic Setup Process

Setting up the LumiraDx Barcode Scanner should take about 5 minutes. Please read through these instructions before starting to setup the scanner.



You will need the following items:

- LumiraDx Barcode Scanner.
- LumiraDx Instrument.
- LumiraDx Test Strip.
- Patient sample ID, if you use these in your institution.

1. Connect the Barcode Scanner to the USB port of the LumiraDx Instrument and wait for 5 seconds. Then scan this barcode:



DEFAULT

2. Listen for a single audible tone and look for the green LED on the Barcode Scanner to flash. Wait for 5 seconds. Then scan the following barcode:



3. Listen for a single audible tone and look for the green LED on the Barcode Scanner to flash. Wait for 5 seconds. Then scan the following barcode:



HOSTCFG 0

4. Listen for a single audible tone and look for the green LED on the scanner to flash. Wait for 5 seconds. Then scan the following barcode:



DLYGDRD 500

5. Repeat section 10.1.1 to check your Barcode Scanner is setup correctly. If you are experiencing problems contact Customer Services.

If you use 2D or more complicated barcodes, you'll need to contact Customer Services to complete the setup process.

10.2 LumiraDx Printer

The LumiraDx Printer can be used to print a Patient or Quality Control (QC) test result onto an adhesive paper.

Patient and QC test results can be printed from the Instrument result page following a test, or from the test details page in the Instrument Result History.

Fully charge the batteries for 16 hours before first use.



10.2.1 Setting up the LumiraDx Printer

Check the paper

The printer comes with a roll of adhesive paper already installed.

Ensure the paper passes through the paper exit slot.



Insert the batteries

Turn the Printer over to see the underside.

Remove the sticker over the battery compartment.

Unclip and lift off the battery cover.

Insert the supplied rechargeable batteries into the Printer paying attention to the '+' and '-', then replace the cover.




Connect the printer

Connect the USB cable and power cable to the Printer.

Fully charge batteries for 16 hours before first use by connecting the Printer to a power source.

Insert the USB connector into one of the USB ports at the rear of the Instrument and press the Printer power button.

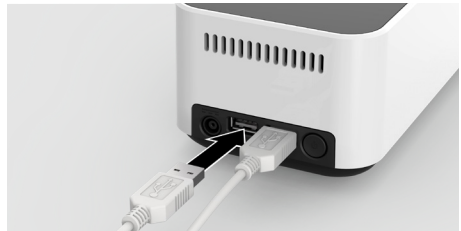

- Take care when transporting to prevent damaging the Printer and batteries.
- Keep away from fire and heating sources.
- Avoid batteries contacting with oxidizers and acid.
- Do not expose the batteries to excessive physical shock or vibration.
- Do not short-circuit.
- Keep batteries out of the reach of children.
- Only use 1.5V NiMH AA cell rechargeable batteries.



10.2.1 Replacing the batteries

Unclip and lift off the battery cover.

If there are already batteries in the Printer, remove the batteries and insert 4 fully charged batteries paying attention to the '+' and '-'. If the batteries are not fully charged, charge for 16 hours before use. Use only 1.5V NiMH AA rechargeable batteries in the Printer.

Align the cover into place and click shut.

10.2.2 Replacing the adhesive paper

Pull the clear central lever on the clear paper cover upwards and forwards until it is release from its locked position. To avoid damage do not use excessive force.

Remove the spool and any remaining adhesive paper. Place a new adhesive paper roll into the paper reservoir, holding approx. 2 inches of the paper outside of the Printer. Ensure the new paper is feeding from the bottom of the roll. Close the cover by applying equal amounts of pressure on each side of the cover until the lid is in the locked position.

Now tear the surplus paper across the serrated edge.

10.2.3 LumiraDx Printer Status light

LED Indication	Condition	Solution
On	Printer On	-
Off	Printer Off	-
* * *	Paper Out	Fit new paper
** ** **	Thermal head too hot	Allow head to cool
*** ** *	Battery cut-out (no charge remaining)	Recharge batteries
**** *****	Battery low (approx. 20% charge remaining)	Recharge batteries

10.2.4 Supplies from LumiraDx

Supplies can be obtained by contacting LumiraDx Customer Services at customerservices@lumiradx.com.


Item	Order code
Adhesive paper rolls	L002120101006

10.3 LumiraDx Connect Hub



Refer to LumiraDx Connect User Manual and Connect Hub Pack Insert for information and setup instructions

10.4 LumiraDx Connect App

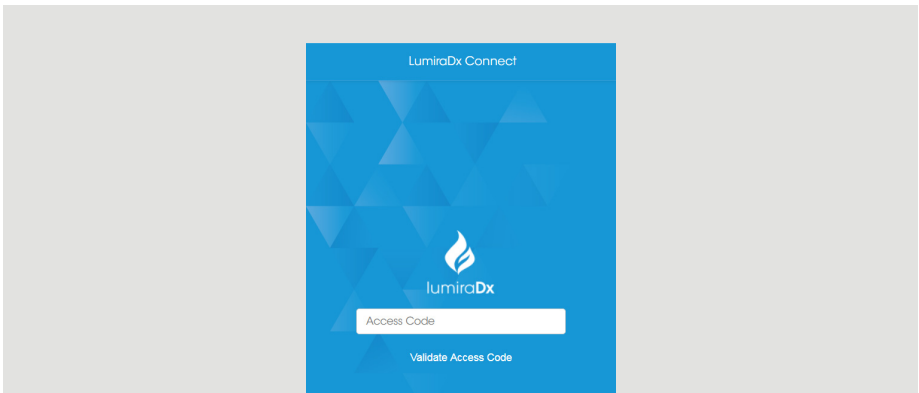

Refer to LumiraDx Connect User Manual for information and setup instructions.

10.5 LumiraDx Platform Instrument Base



LumiraDx Platform Instrument Base used to provide an additional layer of protection for your LumiraDx Instrument by filtering the air that flows into it.

Refer to LumiraDx Instrument Base Pack Insert for further information and instructions.

11 Customer Services

For product inquiries and technical support please contact LumiraDx Customer Services by email: customerservices@lumiradx.com.

For your local customer services phone number please visit the Contact Us page at lumiradx.com.

For further copies of the user manual please contact LumiraDx Customer Services or alternatively an online version can be found at lumiradx.com.

11.1 Warranty

Warranty – Limited Warranty.

LumiraDx Instrument – 2 Year Warranty from date of shipment.

LumiraDx Test Strips – As per shelf life.

For the applicable warranty period, LumiraDx warrants, to the original purchaser only, that each product shall be (i) of good quality and free of material defects, (ii) function in accordance with the material specifications referenced in the Product Insert or Platform User Manual, and (iii) approved by the proper governmental agencies required for the sale of products for their intended use (the "limited warranty"). If the product fails to meet the requirements of the limited warranty, then as customer's sole remedy, LumiraDx shall either repair or replace, at LumiraDx's discretion, the Instrument or Test Strips, as applicable. Except for the limited warranty stated in this section, LumiraDx disclaims any and all warranties, express or implied, including but not limited to, any warranty of merchantability, fitness for a particular purpose and non-infringement regarding the product. LumiraDx's maximum liability with any customer claim shall not exceed the net product price paid by customer. Neither party shall be liable to the other party for special, incidental or consequential damages, including, without limitation, loss of business, profits, data or revenue, even if a party receives notice in advance that these kinds of damages might result.

The limited warranty above shall not apply if the customer has subjected the LumiraDx Instrument or Test Strips to physical abuse, misuse, abnormal use, use inconsistent with the LumiraDx Platform User Manual or Product Insert, fraud, tampering, unusual physical stress, negligence or accidents. Any warranty claim by customer pursuant to the limited warranty shall be made in writing within the applicable limited warranty period.

Details of relevant Intellectual Property regarding this product can be found at lumiradx.com/IP.

Please contact Customer Services for further information about the LumiraDx Instrument third party copyright, privacy notice and general legal statements.

12 References

1. World Health Organization (2009) Guidelines on hand hygiene in healthcare. <https://www.who.int/publications/i/item/9789241597906>
2. World Health Organization (2016) Guidelines on core components of infection prevention programs at the national and acute healthcare facility level. <https://apps.who.int/iris/handle/10665/251730>
3. CDC Guideline for Isolation Precautions: Preventing Transmission of infectious Agents in Healthcare Settings 2007. <https://www.cdc.gov/infectioncontrol/guidelines/isolation/index.html>
4. CDC Clinical Reminder: Use of Fingertick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: <https://www.cdc.gov/injectionsafety/fingertick-devicesbgm.html>

13 Compliance

13.1 Environmental Practices

LumiraDx complies with all environmental legislation in each country where the product is sold. Please refer to lumiradx.com for further information.

13.2 Compliance

LumiraDx UK Ltd hereby declares that the product complies with all relevant European Directives and Regulations at the time of placing on the market. The EC Declaration of Conformity may be found at lumiradx.com. The LumiraDx Instrument has been tested and is compliant with:

CAN/CSA-C22.2 No. 61010-1-12 - Safety requirements for electrical equipment for measurement, control and laboratory use – Part 1: General requirements.

CAN/CSA-C22.2 No. 61010-2-101:15 - Safety requirements for electrical equipment for measurement, control, and laboratory use – Part 2-101: Particular requirements for *in vitro* diagnostic (IVD) medical equipment.

UL 61010-1 (Third Edition) - Safety requirements for electrical equipment for measurement, control and laboratory use – Part 1: General requirements.

UL 61010-2-101 (Second Edition) - Safety requirements for electrical equipment for measurement, control, and laboratory use – Part 2-101: Particular requirements for *in vitro* diagnostic (IVD) medical equipment.

ANATEL – Brazil. This equipment is not entitled to protection against harmful interference and cannot cause interference in duly authorized systems. For more information, access the ANATEL website at www.anatel.gov.br. It incorporates a product approved by Anatel under number (18178-20-13558 and 18181-20-13528) referring to the models OEM-DES-M900-LUM and LDXWS 00946_2.



EU Importer

LumiraDx B.V.
Looskade 20
6041 LE Roermond
The Netherlands



CE mark applies to LumiraDx Instrument,
Test Strips, Quality controls and Connect
Hub only.

LumiraDx Group Limited,
2023 - © All rights reserved, worldwide.



LumiraDx UK Ltd
Dumycal Business Park
Alcoa FK10 2PB, UK



LumiraDx AB
Västra Vågen 5A
16961 Salina
Sweden

SPEC-31530 Rev5
ART-00289 Rev11 Date 2023-07

For product and connectivity support please contact LumiraDx
Customer Services at customerservices@lumiradx.com